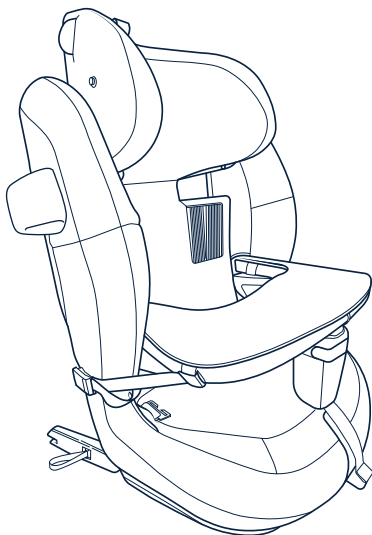
schuchmann SWANDOO

CHARLIE i-Size Booster Sitz

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100 cm – 150 cm

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INTUITIVES SICHERN

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I-SIZE, ECE R129/04

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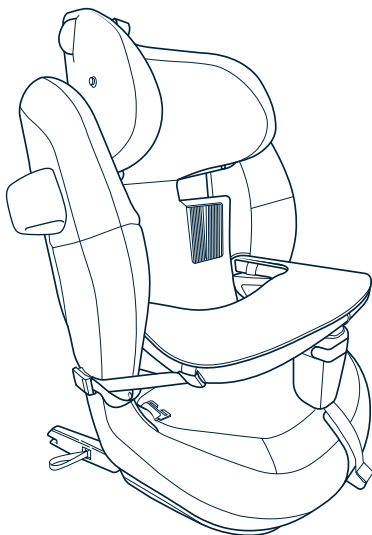
schuchmann SWANDOO

$\int$ CHARLIE i-Size Child Booster Seat

FOR THE BIGGER ONES

100 cm – 150 cm

From approx. 3.5 up to 12 years

INTUITIVE SEAT BELT ROUTING

Safely secured every time

I-SIZE, ECE R129/04

Certified to meet the most demanding
safety regulations

Say 'Hi' to **charlie integral!**
Enjoy the ride!

INDEX

	Installation	30
	Placing the Side Impact Buddy	31
	Securing your child	31
	Removing from the Vehicle	33
	Introduction - charlie integral	33
	Installing charlie integral parts	34
	Washing and Care	39
	Important Information	41
	Warranty	43

PRODUCT PARTS

- A** Adjustable Headrest
- B** Headrest Strap Adjuster
- C** Shoulder Belt Guide
- D** Lap Belt Guide
- E** ISOFIX Sliding Trigger
- F** ISOFIX Connector
- G** ISOFIX Release Puller
- H** ISOFIX Safety Indicator
- I** ISOFIX Guides
- J** Sun Cap Fastener (sold separately)
- K** Fabric Cover
- L** Fabric Cover Installation Tool
- M** User Manual Pocket
- N** User Manual
- O** Side Impact Buddy
- P** Side Impact Buddy Socket

Components for special requirements

- SA** Headrest Padding
- SB** Thorax Pads
- SC** 5-point belt
- SD** Armrests and Armrest Table
- SE** Hip Belt
- SF** Abduction Wedge
- SG** Anatomical Seating Surface
- SH** Footrest

For maximum protection and comfort of your child, it is essential that you read the entire manual carefully and follow all instructions on how to install and use this product. Users with visual or reading impairments must have the instructions for use read out to them.

This is the perfect time to grab a tea or coffee before digging into some important information. Go on! We'll be waiting for you :)

NOTICE

This product has been approved according to the strictest European safety standards (ECE R129/04) and is suitable for children from 100 to 150 cm (from approximately 3.5 to 12 years) in forward-facing direction.

It can only be used for vehicle seats with automatic three-point belts approved in accordance with UN Regulation No. 16 or a comparable standard.

We hereby also declare that the materials used have been tested in accordance with the method in section 5.4 of EN 71-2:2021+A1:2014 have been tested. The seat unit and the individual elements are made of polypropylene, EPP and steel and are corrosion-free. All accessories are padded and covered. The covers are made of 100% polyester.

charlie integral is classified as "i-Size booster seat" for special needs and fits "i-Size seating positions" as indicated by the vehicle manufactures in the vehicles' user manual.

When used for stature heights above 135 cm the compatibility of this seat with your vehicle might be reduced.

Please find the latest version of this user manual online at  **schuchmann.de/produkt/charlie-integral**. Keep the user manual in the back pocket of your car seat. The latest digital version may contain important updates and overrules previous versions.

In case of discrepancy, the latest update should be followed.

Not all configurations of the child seat may be compatible with your vehicle. To find out if the seat fits your car, check **swandoo.com/car-compatibility-list**

BEFORE YOU GO-GO

Please read your vehicle owner's manual for instructions related to child safety seats, and to locate the ISOFIX anchorage points. This seat can be used both with the ISOFIX connectors latched, or unlatched and stowed.

- ❗ The seat must not be used with a two-point seat belt.
- ❗ This child seat must not be used with seats facing perpendicular to the travel direction. The use of this product is permitted on rear-facing seats e.g. in a van or minibus, as long as those are qualified for transporting an adult. The vehicle's headrest must be in place when fitting to a rear-facing seat!
- ❗ The safest place to install this product is in the rear seats of your vehicle.
- ❗ If you need to install this child seat in the front passenger seat, and your car is equipped with Airbags, push the passenger seat as far back as possible. However, make sure the shoulder belt guide of this seat is always in a forward position compared to the B Pillar of the vehicle.
- ❗ Even when not in use, the child seat must always be secured with the vehicle's seat belt.

The restraint function of the child seat remains exclusively with the three-point belt belonging to the vehicle. Belts built into the seat are only used to support and position the child.

INSTALLING IN THE VEHICLE

INSTALLATION WITH ISOFIX

1 If needed place the two ISOFIX guides (**I**) on both ISOFIX anchor points between the seat and the backrest.

2 Extend the ISOFIX connectors (**F**) by pulling the ISOFIX release puller (**E**) in the front of the seat. The ISOFIX connectors will slide out to its most outer position.

3 Position the child seat on the seat in the car and insert both connectors into the ISOFIX anchor points until you hear a "click"

4 Check that the ISOFIX safety indicators (**H**) have turned green.

5 Pull the ISOFIX Sliding Trigger (**E**) again and...

FOR INSTALLATION WITHOUT ISOFIX, START HERE

6 ...push on the child seat bottom part against the vehicle seat until it is firmly positioned and tight to the seat. If the headrest of the vehicle seat is interfering with the child seat, adjust the headrest or remove it completely (do not remove the headrest in rear-facing vehicle seats). The entire back surface of the child integral seat must be in contact with the backrest of the car seat.

7 Pull the three-point belt out and feed the upper part of the belt through the shoulder belt guide (**C**) over the green belt path indicator.

Ensure the belt guide is closed again and that the belt is not twisted.

SIDE IMPACT BUDDY

charlie integral offers high-level integrated side impact protection. The Side Impact Buddy improves the protection even further.

8 Insert the Side Impact Buddy (O) into the red oval socket (P) on the side of the seat.

9 Rotate the Side Impact Buddy 90 degrees until it snaps in.

The Side Impact Buddy always has to be placed on the window side of the child seat and it may not fit or be compatible with every vehicle.

SECURING YOUR CHILD

10 Pull the headrest strap adjuster (B) and adjust the headrest position whilst your child is sitting in the child seat.

11 Move the headrest (A) then slide it upward or downward. Adjust the headrest until the eye indicator matches the eye level of your child or you cannot put about two fingers (approximately 2cm) between the shoulder of your child and the lower edge of the headrest.

12 Pull out the three-point belt, position it in front of the child and lock the belt tongue into the buckle of the vehicle.

- 13** Grab the diagonal belt and pull towards the retractor to remove slack from the lap belt.

! For best protection make sure the belt sits tight and is not twisted. Make sure the lap belt is placed as low as possible on your child's pelvis and not sitting on the abdomen of your child.

! The diagonal part of the belt should sit on your child's collarbone in the centre of the shoulder area and not touch the neck. Adjust the headrest for a better position if necessary.

! Always check the correct installation and restraint of your child before you start driving.

! Do not leave your child unattended in the child seat, not even for a short time.

! Cover the product when exposed to direct sunlight. Parts of the seat may get too hot for your child's skin. Check the temperature before placing your child into the seat.

! Do not use this seat outside of a vehicle.

! Show your child how the belt has to be placed and how to tighten the belt by themselves.

REMOVING FROM THE VEHICLE

- 14** Push the locking mechanism of the shoulder belt guide (C) inwards and slide the three-point belt out of the belt guide.

- 15** Open the ISOFIX connectors (F) by pulling on the ISOFIX Release Puller (G) right on the side of the connectors.

In case the ISOFIX release strap (G) is not accessible use the front puller (E) to move the seat forward.

- 16** Push the ISOFIX connectors back into the seat while pulling on the strap on the front (E). and then remove the seat from the vehicle.

INTRODUCTION - charlie integral.

APPLICATION AREAS AND USE ACCORDING TO THE INTENDED PURPOSE

The charlie integral seat and restraining system is a medical product of risk class 1 and is available in one size. It is designed for the transportation of children and young people with disabilities. It is designed for conveyance of the seat and the equipment with disability-specific accessories is done by the specialist dealer.

INDICATIONS

charlie integral offers support to children who suffer on motor developmental delay (e.g. neuromotor impairments like cerebral palsy, spina bifida, Duchenne muscular dystrophy, spinal muscular dystrophia,...) which lead to involuntary movements, muscular hypo- or hypertonia, weakness, reduced trunk stability, joint deformation among others ...). With special functions and adjustments, charlie integral reduces these symptoms and helps to maintain physiological positioning during transfers. In general, the indication for sitting should be supported by an orthopaedic doctor. It should therefore be clarified whether the patient has any

contraindications before treatment. In general, any type of pain is a contraindication.

INSTALLING THE HEADREST PADDING

! The headrest padding (**SA**) is used to reduce and better adapt the headrest to the head of the child. It also stabilize the child's head.

17 The headrest padding (**SA**) is applied on top of the existing headrest fabric of charlie. Hook the two elastic bands located on the underside of the headrest padding into the hooks under the child seat headrest.

THORAX PADS

! The thorax pads stabilise the child's upper body and are installed on the left and right side. This keeps the child stable and still allows them to move their arms freely and actively.

18 The thorax pad (**SB**) is adjusted using two adjustment wheels. To adjust the width and height, loosen the wheels by turning it to the left and bring the thorax pad into the desired position. Tighten both wheels again and repeat the process on the other side if necessary.

5-POINT BELT

! The 5-point harness can be used help to stabilise the child's upper body, however it is not a restraint system. Accordingly, the 3-point belt of the car must be used, and must be located on top of the 5-point harness.

! This "Special Needs Restraint" is designed to give extra support to children who have difficulty sitting correctly in conventional seats. Always consult your doctor to make sure that this restraint system is suitable for your child.

19 Take the 5-Point-Belt (SC). Attach the two loops in the black metal brackets on the right and left below the headrest. Attach the other two loops on the other ends of the belt in the black metal hooks on the right and left of the seating area.

Attach the loop underneath the central buckle in the black metal bracket that protrudes from the center of the seat area. Adjust the length of the buckle strap to ensure the best comfort for your child.

Place the crotch pad around the central buckle of the harness.

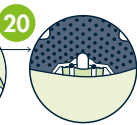
Fasten your child and adjust the length of both sides of the harness until they touch the child's chest.

After placing the 5 point harness on the child, continue with restraining the child with the 3 point vehicle belt as shown in chapter **SECURING YOUR CHILD**.

HIP BELT

20 Insert the metal part of the Hip Belt (SE) into the opening in the middle of the fabric cover in the lower back area.

Ensure that the metal part is positioned in the fabric loop behind the fabric cover.

Push the Hip Belt upwards until the metal part is fully inserted and the belt adheres to the backrest of the seat.

Adjust the length of the belt on both sides to ensure a comfortable sitting position for your child. If you shorten the length of the belt, you may need to remove the side pads.

HIP BELT WITH 5-POINT-BELT

21 The pelvic belt can also be used in combination with the 5-point harness. Attach the hip belt as described in the chapter **HIP BELT (20)**

Install the 5-point harness as described in the chapter **(19) 5-POINT HARNESS.**

Before attaching the lower strap of the 5-Point- Belt (**SC**) to the black metal bracket, insert it through the fabric loop on the Hip Belt (**SE**).

Fasten your child and adjust the length of both sides of the harness until they touch the child's chest.

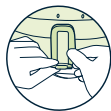
ARMRESTS AND ARMREST TABLE

! The arm support pad supports children who are unable to hold themselves upright. It helps to maintain trunk and head control and supports trunk stability.

Clamps are attached to both sides of the car seat. The support pad is attached there using a buckle. The arm support cushion also serves to guide the thighs.

22 Attach the two armrest (**SD**), as shown in the images.

Attach the flap in the front of the seating area by inserting it under the seating shell and fabric cover with the velcro pointing forward.

Place the Armrest Table (**SD**) on top of the Armrests. Adjust the height of the armrest leg as needed by adjusting the strap that runs around its top.

Attach the gray strap with the velcro to the flap that was previously attached to the front of the seat.

The straps on both sides of the armrest table (SD) can now be attached to both sides of the car seat with magnetic buttons. Both straps can be pulled tight with the attached loops.

ABDUCTION WEDGE

! The abduction wedge (SF) is used to improve the positioning of the legs. Abduction wedges are designed to keep the thighs apart and ensure a certain degree of hip abduction. They are used to prevent malpositions in children or for post-operative rehabilitation of the hip joint.

23 To attach the abduction wedge (SF), insert the two metal rods into the holes in the front part of the seat. Pull the elastic band of the abduction wedge and fasten it inside the belt at the front of the seat using the snap fastener.

ANATOMICAL SEATING SURFACE

24 The anatomical seating surface is placed under the fabric in the seating area.

Open the velcro on the back of the seat and pull the fabric up on the sides of the seating surface.

Remove the existing cushioning layer as well as the seating surface in the back of the seating area.

! In case your child needs a special seating surface, please contact Schuchmann for available options.

Tuck the fabric back into place with the fabric tool behind the headrest and close the velcro in the back of the seat.

FOOTREST

! The footrest helps children to straighten up independently. The sole of the foot resonance calms the children by providing feedback in the event of perceptual disturbances. It also helps to keep the lower extremities in physiological alignment to reduce pathological movement patterns and facilitate weight bearing.

! The footrest is not to be used as a step for getting in and out of the seat independently. Do not allow your child to stand on the footrest! The footrest needs to be installed before installing the seat.

25 Place the footrest (**SJ**) in your vehicle before installing the child seat as shown in the image.

If the seat on which you are fitting the footrest is equipped with ISOFIX attachment, hook the hooks into the anchorage points firstly, and then install the ISOFIX attachment into the anchorage point.

If the seat is not equipped with ISOFIX connections: Open the harness straps, remove the hooks and replace them with the anchoring plates that you will find

in the enclosed envelope. Push these through as far as they will go between the backrest and the seat cushion of the seat.

Turn the gray rotary knob anticlockwise. Pull the foot of the leg out until it reaches the floor of the vehicle.

Once the height is defined, secure the position by tightening the knob clockwise.

Tighten the straps on both sides and place the child seat on top of the mounting plate.

Adjust the height of the footrest platform to the comfort needs of your child.

WASHING AND CARE

REMOVING THE FABRICS FOR WASHING

Remove the fabric from the headrest by simply lifting it up from the edges. Open the fabrics on the back of the child seat. To prevent the cover from slipping forwards whilst in use, there are two small hooks beneath the cover holding it in place.

Ensure the cover is detached from these hooks by pinching the cover at each side of the seating area and pulling towards the centre of the seat prior to removing the fabric completely. Loosen the whole cover by lifting it up from the edges.

REPOSITIONING OF THE FABRICS

Unless already removed for washing, you'll find the fabric installation tool behind the headrest. Tuck the fabric edge into the right position. Ensure the cover is reinstalled the same way they previously were. You can follow the previous steps in reverse order to make sure everything is in place.

WASHING INSTRUCTIONS

Please wash before first use!

Machine washable: Fine/delicate programme.

Hand wash recommended.

Use mild detergents. Do not iron, dry clean, bleach or tumble dry. Keep away from your dog, your cat and direct heat. Do not use bleach, abrasive, upholstery or aggressive cleaning agents.

Clean the plastic parts with a mild detergent and warm water. Do not use any lubricants.

! The fabric cover is a safety feature and the child seat should never be used without it.

If the cover needs replacing at any time, only use official Swandoo charlie integral covers.

DISINFECTION

Various products can be used to disinfect the surfaces of metal and plastic parts. Liquid disinfectants are available as a ready-to-use solution that is sprayed on and applied evenly with a soft cloth. Alternatively, cloths pre-soaked with disinfectant can be used to wipe the products over the entire surface. In both cases, care must be taken to ensure complete wetting. Disinfection in fully automatic disinfection systems is also possible and recommended. The exposure time may vary and can be found in the manufacturer's instructions for the agents used.

MAINTENANCE

Please carry out a daily visual inspection and check the product regularly for cracks, breaks, missing parts and malfunctions. In the event of a defect or malfunction, please contact the supplying specialist dealer directly.

MAINTENANCE

For reasons of user safety and to maintain product liability, maintenance must be carried out at least once a year by a specialist dealer. The maintenance work carried out must be documented in the maintenance schedule.

MAINTENANCE REQUIREMENTS

Basic cleaning according to the manufacturer's instructions. Disinfection according to the manufacturer's instructions if necessary. Damage to the frame, add-on parts and accessories (cracks, breaks, corrosion, bent or missing parts). Strength of the connections (tighten loose screws, replace missing screws). Functionality of the adjustment elements (screws, release lever). Functionality of other adjustment elements (backrest, headrest, pads, footrests and guides in the leg area).

Functionality of the safety elements (ISOFIX brackets). Checking the harness for damage (clamping devices, fasteners, seams). Checking the upholstery and covers for damage. Legibility of the type plate. Final complete functional check of the aid. Checking the correct attachment of all add-on parts and the associated accessories.

MAINTENANCE SCHEDULE

Maintenance specifications of the manufacturer were carried out:

Date	Company	Name	Signature

PRODUCT LIFETIME AND REUSE

The expected service life of our product, depending on the intensity of use, is up to "8" years if used in accordance with the instructions in this manual. The product can be used beyond this period if it is in a safe condition. The expected service life does not apply to wearing parts such as covers, upholstery, etc. Schuchmann excludes the purchase or sale of used child seats, as these may have invisible damage which can significantly reduce the safety of the child seat.

 The charlie integral car seat is not suitable for reuse!

IMPORTANT INFORMATION

Never use the product in any configuration or assembly method not shown in the instructions.

Only use accessories and spare parts from Schuchmann,

otherwise you may endanger the safety of the user and operator.

Do not use any load bearing contact points other than those described in the instructions and marked on the child restraint.

Make sure luggage or other objects are properly secured. Unsecured luggage may cause severe injuries to children and adults in an accident.

Only adults should fix or adjust the product. Do not make any changes to the product. This could affect its performance and make it unsafe.

Do not use the product with damaged or missing parts.

Do not use a second-hand child seat or child seat whose history you do not know. It is dangerous to use unauthorized or changed products!

Do not use the product if there is any evidence of unlicensed modification.

Do not remove any labels on the product. They contain important information.

Your Swandoo charlie integral must undergo regular maintenance.

Never use the seat as a means of transportation other than its intended purpose.

In an emergency, it is important that you can quickly release the child from the seat. For this reason, the belt buckle is not completely secured against tampering. The child should therefore never play with the buckle.

This product should be replaced when it has been subject to violent stress in an accident.

To preserve and protect the environment, to prevent pollution and to improve the reuse of raw materials (recycling), please observe the following disposal instructions: The packaging of the product should be kept for possible future transportation. If you need to return the product to us for repair or in the event of a warranty claim, please use the original carton if

possible so that the product is optimally packaged. Otherwise, recycle the packaging materials according to their type.

! Do not leave the packaging materials unattended, as they are potential sources of danger. At the end of the product life cycle, recycle the raw materials used for the product according to their type.

Ensure that the rigid items and plastic parts of an Enhanced Child Restraint System are located and installed so that they are not likely to become trapped by a movable seat or in a door of the vehicle during everyday use.

NOTE

If a serious incident occurs during the service life of the product during its intended use, this must be reported immediately to the manufacturer and the responsible authority.

WARRANTY

The two-year statutory warranty period applies to all products. This begins with the delivery or handover of the goods. If a material or manufacturing defect demonstrably occurs in the goods delivered by us within this period, we will inspect the reported damage and, if necessary, repair or replace the goods free of charge and at our discretion, provided the goods are returned to us carriage paid.

If you have any questions or concerns about this product or this user manual, please contact us before using the product. We look forward to hearing from you!

We look forward to hearing from you!

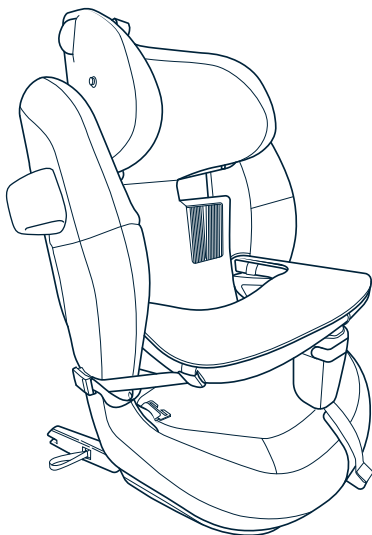
If you have any questions, please contact our team.
We look forward to hearing from you!

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© Swandoo 2025, All rights reserved
Swandoo GmbH