

momo therapy bicycle.



Instructions for use.

momo therapy bicycle. The bicycle for more balance.




schuchmann®

Many thanks.



Dear Customer

At this point we would like to thank you for placing your trust in our company and for purchasing our product. We ask you to read through the Instructions for use carefully prior to initial commissioning of the product, and to observe them. Please note that guidelines and representations in these Instructions for use may deviate from your product due to differing equipment. We reserve the right to make technical modifications.

Important information!

Ensure that these Instructions for use remain with the product.

Your **schuchmann** Team



1. Preparation.	05
1.1 Delivery.....	05
1.2 Safety measures prior to use	05
1.3 Safe disposal.....	05
1.3.1 Packaging.....	05
1.3.2 Product.....	05
1.4 Where to store the Instructions for use.....	06
2. Product description.	06
2.1 General information.....	06
2.2 Handling and transport.....	06
2.3 Application areas, use according to the intended purpose	06
2.3.1 Indications.....	07
2.3.2 Contraindications	07
2.4 Use not in accordance with the intended purpose / warning guidelines.....	08
2.5 Equipment for basic model.....	09
2.6 Product overview.....	09
2.7 Overview of equipment / accessories.....	10
2.8 Drive possibilities	16
2.9 Adjusting the stabilisers	17
2.10 The initial riding attempts.....	17
3. Settings.	18
3.1 Presettings	18
3.1.1 Handlebar adjustment	18
3.1.2 Saddle adjustment	19
3.2 Brakes.....	20
3.2.1 Parking brake	20
3.2.2 Round handlebars with brake lever ring	21
3.2.3 Round handlebars with brake function	21
3.2.4 Drum brake in front wheel	21
3.2.5 Backpedal brake.....	21
3.3 Tyres and hoses.....	22
3.4 Chain tension.....	22
3.5 Light system / dynamo	22
4. Accessories.	23
4.1 Dynamic back and pelvic guide pelotte pads	23
4.1.1 Width adjustable back and pelvic guide pelotte pads.....	23
4.2 Headrest.....	23
4.3 Push bar.....	24
4.4 Parking brake for accompanying escorts	24
4.5 Steering for accompanying persons.....	24
4.6 Handlebar lock limiter	25
4.7 Holding bracket with mount.....	25
4.8 Universal adapter.....	26
4.9 Crank shortener (continuously adjustable)	26
4.10 Crank shortener	26
4.11 Special crank for knee contracture.....	27



5. Foot positioner.	27
5.1 Exercise bike pedals	27
5.2 Foot positioning pedals.....	27
5.3 Foot pans.....	28
5.4 Foot pans with leg guidance.....	29
5.5 Foot pans with dynamic leg guidance.....	29
6. Strap systems.	30
6.1 Chest strap.....	30
6.2 Positioning vest.....	30
6.3 Groin harness, T-shaped.....	31
6.4 4-point pelvic harness	31
6.5 Hand positioning aid.....	31
7. Electric drives.	32
7.1 E-drive.....	32
7.2 Light system.....	33
8. Cleaning and servicing.	34
8.1 Cleaning and disinfecting.....	34
8.1.1 Cleaning.....	34
8.1.2 Disinfection.....	34
8.1.3 Chain maintenance.....	35
8.2 Servicing / controls.....	35
8.3 Maintenance	35
8.3.1 Maintenance specifications.....	35
8.3.2 Maintenance plan	36
8.4 Repairs.....	37
8.5 Controls.....	37
8.6 Storage	37
8.7 Spare parts.....	38
8.8 Duration of use and re-use	38
8.9 Error messages and troubleshooting	39
9. Technical data.	43
9.1 Dimensions.....	43
9.2 Tyre pressure.....	43
9.3 Torque information.....	43
9.4 E-drive.....	44
9.5 Lighting system.....	45
10. Guarantee.	45
11. Identification.	46
11.1 Serial number / date of manufacture	46
11.2 Product version.....	46
11.3 Issue of the document.....	46
11.4 Name and address of the manufacturer, specialist dealer supplying the product.....	46

1. Preparation.

1.1 Delivery

On receiving the product, please check it for completeness, lack of faults and any transport damage. Inspect the goods in the presence of your forwarder. Should transport damage have occurred, please arrange for an inventory (determination of the faults) to be made in the presence of the forwarder. Please send a complaint in writing to the specialist dealer responsible.

1.2 Safety measures prior to use

The correct use of the product requires detailed and thorough instruction of the user or the accompanying person. We ask you to read through the Instructions for use carefully prior to initial commissioning of the product, and to observe them. Screws may have loosened during transport. Before initial operation, ensure that all screws are properly tightened to the torque values specified on the respective components, if applicable. It is possible that product parts that get in contact with the skin may heat up in the sun. Depending on the duration and intensity of solar radiation, the surfaces of individual parts can heat up to over 41°C and thus lead to slight burns in the event of direct skin contact. Therefore cover these areas or protect the device from direct sunlight.

1.3 Safe disposal

In order to preserve and protect the environment to prevent environmental pollution and to improve the recycling of raw materials, please note the disposal instructions in points **1.3.1** and **1.3.2**.

1.3.1 Packaging

The product packaging should be stored in case the product needs to be transported again. If you need to return the product to us for repairs or in the event of warranty claims, kindly use, if possible, the original carton box so that the product is optimally packaged. Recycle the packaging materials according to their type.



Do not leave packaging materials unattended, as they are a possible source of danger.

1.3.2 Product

Separate the raw materials used in the product for recycling according to their classification (see material information under **Point 2.1**). The electrical components and the drive system battery must not be disposed of with household waste. These must be returned to the designated public collection points or to your specialist dealer (see **Point 9.5**).



1. Preparation.

1.4 Where to store the Instructions for use

Please store these Instructions for use carefully and ensure that these Instructions for use remain with the product in case of re-use. Should you lose the instructions, you can always download an updated version at www.schuchmann.de.

2. Product description.

2.1 General information

All base frames are made of aluminium, which is non-corroding and powder-coated. All other materials used are protected against corrosion through the use of stainless steel, aluminium or plastic. All important parts, such as the saddle, handlebars or also the pedals, can be individually adapted to the individual requirements. Thanks to special accessories the user can be positioned e.g. at the torso or in the calf area / foot area. The tricycles generally have a brake hub / backpedal hub brake on the rear wheel (except for those with a rigid sprocket) and an air pressure-independent parking brake on the front wheel.

2.2 Handling and transport

The **momo therapy bicycle** is not intended for carrying since it is equipped with tyres. Should you have to carry the equipment due to obstacles, ensure that all moving parts are tightened. Subsequently, stand in twos on either side of the therapy bicycle, hold the frame from the left and right and carry it to the desired location. In order to transport the therapy bicycle, reduce all adjustments to their most compact size (saddle height, handlebar height, remove accessories etc.).

2.3 Application areas, use according to the intended purpose

The **momo therapy bicycle** is a class I medical device and was designed for children, adolescents and adults for outdoor use.

The tricycle enables users with disabilities to move around independently. They can use the tricycle to cover relevant distances independently and achieve an appropriate speed. The use of the tricycle ensures mobility outdoors and participation in community life (integration into a peer group). Any other use or use beyond this purpose shall be considered not in accordance with the intended purpose.

The motor skills in relation to

- movement coordination (alternating leg movements, eye-hand coordination, isolated arm and leg movements);

2. Product description.

- targeted development of muscle strength and endurance;
- trunk stability and head control;
- the balance reaction is particularly strongly promoted when using the therapy bicycle;
- joint mobility (range of motion, ROM), especially of the lower extremities;

and the patient's cardiorespiratory function are strengthened. These effects support physiotherapy treatment.

2.3.1 Indications

The **momo therapy bicycle** is suitable for children, adolescents and adults with acquired or congenital disabilities due to the following diagnoses:

- Diseases of the central nervous system (e.g. cerebral palsy in spastic, dyskinetic or ataxic form, GMFCS II-IV)
- Diseases of the peripheral nervous system (e.g. such as spina bifida or paraplegia)
- neuro-degenerative diseases (e.g. muscular dystrophies or atrophies caused by metabolic defects)
- genetic diseases (e.g. trisomy 21, Rett syndrome, Kabuki syndrome, Cornelia de Lange syndrome, ...)
- Orthopaedic conditions and resulting deformities (e.g. rheumatism or consequences of trauma to the bony apparatus)
- Mental illnesses that influence movement behaviour (e.g. from the autistic spectrum)

Relevant symptoms (also with other diagnoses) are

- restricted postural control,
- restrictions in the range of movement,
- deformities of the bony apparatus (e.g. spine and/or pelvis),
- restrictions of the cardiorespiratory system,
- impaired balance and equilibrium,
- limitations in strength and endurance,
- uncontrollable movement.

The group of people described cannot use standard bicycles or vehicles, even with standard training wheels, due to the movement disorders.

2.3.2 Contraindications

In general, the indication should be accompanied by a doctor/orthopaedist. It should therefore be clarified prior to procurement whether contraindications exist for the patient. Any danger to the user or others through the use of a bicycle with support wheels should be excluded. In general, any type of pain represents a contraindication.



2. Product description.



2.4 Use not in accordance with the intended purpose / warning guidelines

- Replace bent handlebars and handlebar stems immediately! Breakage may occur in the event of continued use or repairs.
- Correct usage of the product requires precise and careful training of the accompanying person.
- The vehicle may only be used on stable and flat ground.
- Please observe the "Technical data" in these instructions for use for the maximum permitted patient weight.
- Always wear light-coloured and distinctive clothing!
- Always be ready to brake, in particular in steep terrain and sections which are not easy to assess!
- Show consideration for other people who are walking or hiking!
- Do not hang loads on the handlebars; this compromises the travel safety.
- Test the fastenings for the pedal cranks and pedals regularly!
- For your own safety, we recommend that you always use your vehicle with a helmet. Please ensure in particular that the helmet is of good quality. It should accord at least with the legal regulations or recommendations (Standard: EN 1078 or ANS)!
- Check that the brakes, lights and bell function properly prior to each journey!
- Check all screw connections and, if necessary, retighten them using the specified torque values.
- Ensure that your vehicle accords with the legal requirements!
- Only ride if the bicycle is in proper condition!
- Do not use headphones so that you can remain aware of warning sounds.
- If you mount a push rod, it must only be used for guiding the therapy bicycle. The push not is NOT suited for relocating the therapy bicycle or for lifting or tilting!
- In wet conditions, the braking distance of your bicycle will become longer. Therefore ensure to always set your speed in such a manner that you can stop at any time.
- The therapy bicycle is not suitable for carrying a second person. The consequences arising from such use, which is not in accordance with the intended purpose, shall not fall within the scope of responsibility of the manufacturer.
- The bicycle basket may only bear loads of up to 20 kg.
- When adjusting the therapy bicycle there is the risk of jamming or crushing extremities.
- Users who have difficulty reading must have someone read these Instructions for use aloud so that they understand how to use the standing exercise trainer.

2. Product description.

2.5 Equipment for basic model

- Aluminium frame with extra-low access point
- Rim brake with separate parking brake
- Angle adjustable handlebar stem
- Handlebar damper for stabilisation when moving straight on
- Drum brake in front wheel
- Stabiliser frame with dynamically-working stabilisers and adjustable spring strength
- Luggage carrier

2.6 Product overview

The Fig. below is intended to show you the designation of the most important components as well as the terms which you will find in these Instructions for use.

**Dynamic back pelotte pad
incl. chest strap**

**Dynamic brake pelotte pad
incl. chest strap**

Saddle

Luggage carrier

**Stabiliser
frame**

Stabiliser

Pedal

Handlebars

Bell

Locking brake

Drum brake

**Aluminium frame with
extra low access**



2. Product description.

2.7 Overview of equipment / accessories

Saddle						
Art. No.		Saddle			Width	Length
3701001		Standard saddle Size 1			15 cm	21 cm
3702001		Standard saddle Size 2			15 cm	24 cm
3703001		Standard saddle Size 3			18 cm	26 cm
Art. No.		Saddle	Rear width	Front width	Length	
3701024		Gel saddle Size 1	19 cm	4 cm	24 cm	
3702024		Gel saddle Size 2	24 cm	7 cm	27 cm	
Art. No.		Saddle	Rear width	Front width	Rear length	Length total
3701003		Saddle seat with ischium depressions Size 1	20.5 cm	4 cm	9.5 cm	14.5 cm
3702003		Saddle seat with ischium depressions Size 2	24 cm	4 cm	9.5 cm	14.5 cm
Art. No.		Saddle	Rear width	Width in centre	Front width	Length
3700010		Unicycle saddle – Banana-shaped (Step length increases by 2 cm)	11 cm	6 cm	8 cm	25 cm
Art. No.		Saddle	Rear width	Front width	Rear length	Length total
3700023		Moped saddle (Inside leg length increases by 2 cm)	26 cm	9.5 cm	12 cm	25 cm

Saddle posts		
Art. No.	Saddle post	
3701004		Standard saddle post Size 1 for 12"
3702004		Standard saddle post Size 2 for 16"
3703004		Standard saddle post Size 3 for 20" - 26"
Art. No.	T-saddle post	
3701005		T-saddle post Size 1 for 12"
3702005		T-saddle post Size 1 for 12"
3703005		T-saddle post Size 3 for 24" + 26"

Holding bracket				
Art. No.	Holding bracket with mount		Max. height*	Depth
37 01 007		Holder bracket Size 1	20 cm	12 cm
37 02 007		Holder bracket Size 2	30 cm	12 cm
37 03 007		Holder bracket Size 3	37 cm	12 cm
37 04 007		Holder bracket Size 4	53 cm	12 cm
37 09 007		Holder bracket, customer-made	___ cm	___ cm
37 02 055		Universal bracket – for harnesses when not using pelotte pads (width = 27.5 cm)		
* Max. height: measured on a standard saddle up to the top edge of the back pelotte pad				

* Max. height: measured on a standard saddle up to the top edge of the back pelotte pad

2. Product description.

Headrest				
Art. No.		Headrest	Width	Height
3701029		Headrest Size 1	20 cm	15 cm
3702029		Headrest Size 2	23 cm	18 cm

Handlebars				
Art. No.		Classic handlebars – black	Width	
3701011		Classic handlebars for 12"	47 cm	
3702011		Classic handlebars for 16"	49 cm	
3703011		Classic handlebars for 20"	52 cm	
3704011		Classic handlebars for 24"-26"	58 cm	
Art. No.		Touring handlebars	Width	Depth
3701012		Touring handlebars Size 1	55 cm	17 cm
3702012		Touring handlebars Size 2	58 cm	17 cm
Art. No.		Round handlebars	Width	Depth
3701013		Round handlebars Size 1	40 cm	17 cm
3702013		Round handlebars Size 2	43 cm	21 cm
3703013		Round handlebars Size 3	48 cm	25 cm
Art. No.		Multifunctional handlebars – suitable for 20" – 26"	Width	Depth
3702014		Multifunctional handlebars	61.5 cm	16.5 cm
Art. No.		Handlebar lock limiter – adjustable up to direction determination		
3701006		Handlebar lock limiter Size 1 for 12"		
3703006		Handlebar lock limiter Size 2 for 16" – 26"		
Art. No.		Handlebar extension	Length	
3701022		Handlebar extension Size 1 for 12"	10 cm	
3702022		Handlebar extension Size 2 for 16" – 26"	10 cm	

Foot pans									
Art. No.		Foot pans with leg guidance	Min. width at front	Max. Front width	Min. width at rear	Max. Rear width	Length	Min. height of leg guidance	Max. height Leg guidance
3701018		Size 1	8.7 cm	10.4 cm	5.7 cm	7.4 cm	17.4 cm	15 cm	18.5 cm
3702018		Size 2	9.5 cm	12 cm	6.7 cm	9.2 cm	20.1 cm	18 cm	22 cm
3703018		Size 3	11.5 cm	14 cm	8.1 cm	10.6 cm	23.8 cm	21 cm	26 cm
3704018		Size 4	11.5 cm	14 cm	8.1 cm	10.6 cm	23.8 cm	28 cm	36 cm
Art. No.		Foot pans			Min. width at front	Max. width at front	Min. width at rear	Max. width at rear	Length
3701017		Foot pans Size 1			8.7 cm	10.4 cm	5.7 cm	7.4 cm	17.4 cm
3702017		Foot pans Size 2			9.5 cm	12 cm	6.7 cm	9.2 cm	20.1 cm
3703017		Foot pans Size 3			11.5 cm	14 cm	8.1 cm	10.6 cm	23.8 cm



2. Product description.

Foot pans									
Art. No.		Foot pans with dynamic leg guidance	Min. width at front	Max. width at front	Min. width at rear	Max. width at rear	Length	Min. height of leg guidance	Max. height of leg guidance
37 01 035		Size 1	8.7 cm	10.4 cm	5.7 cm	7.4 cm	17.4 cm	15 cm	18.5 cm
37 02 035		Size 2	9.5 cm	12 cm	6.7 cm	9.2 cm	20.1 cm	18 cm	22 cm
37 03 035		Size 3	12 cm	15 cm	8.8 cm	11.8 cm	24 cm	20.5 cm	25.5 cm

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
Pedals			
Art. No.		Pedals	Depth
3700013		Foot positioning pedals - with bike toe clips and compensation weights	13 cm
Art. No.		Pedals	Width
3700014		Exercise bike pedals	12 cm

Back pelotte pads			
Art. No.		Dynamic back pelotte pads – padded with strap and mount	Width
37 01 008		Dynamic back pelotte pad Size 1	20 - 30 cm
37 02 008		Dynamic back pelotte pad Size 2	25 - 35 cm
37 03 008		Dynamic back pelotte pad Size 3	30 - 40 cm
Art. No.		Width adjustable back pelotte pad - padded, including mount	Width
37 01 025		Width adjustable back pelotte pad Size 1	20 - 28 cm
37 02 025		Width adjustable back pelotte pad Size 2	25 - 33 cm
37 03 025		Width adjustable back pelotte pad Size 3	30 - 36 cm

Pelvic supports			
Art. No.		Dynamic pelvic guidance pelotte pad – padded, with strap and mount	Width
3701009		Dynamic pelvic guidance pelotte pad Size 1	20 – 30 cm
3702009		Dynamic pelvic guidance pelotte pad Size 2	25 – 35 cm
3703009		Dynamic pelvic guidance pelotte pad Size 3	30 – 40 cm
Art. No.		Width adjustable back guidance pelotte pad - padded, including mount	Width
3701026		Width adjustable pelvic guidance pelotte pad Size 1	20 – 28 cm
3702026		Width adjustable pelvic guidance pelotte pad Size 2	25 – 33 cm
3703026		Width adjustable pelvic guidance pelotte pad Size 3	30 – 36 cm

2. Product description.

Positioning						
Art. No.	Hand positioning aid – for all handlebars					Front width
3700054		Hand positioning aid size 0				8.5 cm
3701054		Hand positioning aid size 1				10 cm
3702054		Hand positioning aid size 2				12 cm
3703054		Hand positioning aid size 3				14 cm
Art. No.	Chest strap – for the width adjustable back pelotte pad			Width	Length	
3701050		Chest strap Size 1		5 cm	18.5 cm	
3702050		Chest strap Size 2		6.5 cm	25.5 cm	
3703050		Chest strap Size 3		7 cm	30.5 cm	
Art. No.	Positioning vest – for the width adjustable back pelotte pad		Inside width	Total width	Side height	Total height
3701051		Positioning vest size 1		8 cm	25 cm	10 cm
3702051		Positioning vest size 2		9 cm	30 cm	12 cm
3703051		Positioning vest size 3		12 cm	34 cm	16 cm
Art. No.	Groin strap, T-shaped – for the width adjustable pelvic pelotte pad		Width Seat surface	Front width	Length Seat surface	Total length
3701052		Groin harness, T-shaped Size 1		24 cm	23 cm	15 cm
3702052		Groin harness, T-shaped Size 2		26 cm	26 cm	17 cm
3703052		Groin harness, T-shaped Size 3		28 cm	28 cm	19 cm
Art. No.	4-point pelvic harness – for the width adjustable pelvic pelotte pad		Width Exterior parts	Width Centre section	Length Exterior parts	Length Centre section
3701053		4-point pelvic harness Size 1		7 cm	7 cm	12 cm
3702053		4-point pelvic harness Size 2		9 cm	8.5 cm	13 cm
3703053		4-point pelvic harness Size 3		11.5 cm	11 cm	15 cm

Handlebars		
Art. No.	Push bar – height adjustable, removable and equipped with an integrated anti-rotation lock	
3801009		Push rod for 12"
3802009		Push rod for 16"
3803009		Push rod for 20 – 26"
Art. No.	Parking brake for accompanying persons – only usable in connection with push bar	
3700012		Parking brake for accompanying persons
Art. No.	Steering for accompanying person using steering linkage including handlebar lock limiter (can only be used in connection with push bar)	
3801010		Steering for accompanying person for 12"
3802010		Steering for accompanying person for 16"
3803010		Steering for accompanying person for 20"



2. Product description.

Light system		
Art. No.	Light system	
38 01 021		Retrofitting of a lights system for 16" + 20" with side dynamos and reflectors acc. German Road Traffic Act Regulations
38 02 021		Retrofitting of a lights system for 16" + 20" with hub dynamo and reflectors acc. StVZO (German Road Traffic Act Regulations)
38 05 021		Light system when using an electric drive for 16" + 20"
38 03 021		Lights system with hub dynamo for 24" + 26"

Basket		
Art. No.	Basket	
38 01 019		Basket for 12" + 16"
38 03 019		Basket for 20" + 24"
38 05 019		Basket for 26"

Handlebar brake			
Art. No.	Handlebar brake		for
37 01 028		Round handlebars with brake lever ring	Size 1
37 02 028			Size 2
37 01 031		Round handlebars with brake function The actuation of the braking function takes place by simply pressing down the round handlebars	Size 1
37 02 031			Size 2

Conversion kit		
Art. No.		for
38 01 026	Quick connection of the stabiliser frame for easy disassembly for transport	For 12"
38 02 026		for 16-26"
38 01 101	Stabiliser frame with dynamically-working stabilisers and adjustable spring strength	For 12"
38 02 101		For 16"
38 03 101		For 20"
38 04 101		For 24"
38 05 101		For 26"
38 00 025	Assembly at Schuchmann	

2. Product description.

DirectPower E-drive from Heinzmann

The **momo therapy bicycle**. can optionally be equipped with the DirectPower E-drive from Heinzmann. This allows us to offer you a wear-free and silent direct drive. The motor is designed to be brushless and without gears. This minimises maintenance and wear and eliminates motor noise. The recuperative system automatically recharges when driving downhill or braking. This means that the range can be increased by up to 15 %. The existing force of the user is measured by a force sensor integrated in the bottom bracket, and the assistance from the electric drive is adjusted.

You have selected one of the following e-drive configurations (with or without start-up assistance) for the **momo therapy bicycle**. Here you have chosen a **beginner version** or an **advanced version**. The difference here is the **start-up** and the **top speed** of the various modes **Eco, Standard and Power**, which can be selected via the display. Using an analysis cable, the chosen version can be changed by our field service team later on.

DirectPower E-drive from Heinzmann – without start-up assistance

for size	Revision		Start-up handling / top speed		
			Eco	Standard	Power
16"	Beginner	(36 02 070)	gentle / 4 km/h	medium / 4 km/h	strong / 6 km/h
	Advanced	(36 02 071)	medium / 6 km/h	strong / 8 km/h	strong / 10 km/h
20"	Beginner	(36 03 070)	gentle / 4 km/h	medium / 4 km/h	strong / 6 km/h
	Advanced	(36 03 071)	medium / 6 km/h	strong / 8 km/h	strong / 12 km/h
24"	Beginner	(36 04 070)	gentle / 6 km/h	medium / 8 km/h	strong / 12 km/h
	Advanced	(36 04 071)	gentle / 10 km/h	medium / 15 km/h	strong / 20 km/h
26"	Beginner	(36 05 070)	gentle / 6 km/h	medium / 8 km/h	strong / 12 km/h
	Advanced	(36 05 071)	gentle / 10 km/h	medium / 15 km/h	strong / 20 km/h

DirectPower E-drive from Heinzmann – with start-up assistance

The start-up assistance can be activated depending on the chosen version at the touch of a button (fig. 1) or using a twist grip (fig. 2).



for size	Revision		Start-up handling / top speed			
			Start-up assistance	Eco	Standard	Power
16"	Beginner with twist grip	(36 02 072)	4 km/h	mild / 4 km/h	medium / 4 km/h	strong / 6 km/h
	Beginner with button	(36 02 073)				
	Advanced with twist grip	(36 02 074)	6 km/h	medium / 6 km/h	strong / 8 km/h	strong / 10 km/h
	Advanced with twist grip	(36 02 075)				
20"	Beginner with twist grip	(36 03 072)	4 km/h	mild / 4 km/h	medium / 4 km/h	strong / 6 km/h
	Beginner with button	(36 03 073)				
	Advanced with twist grip	(36 03 074)	6 km/h	medium / 6 km/h	strong / 8 km/h	strong / 12 km/h
	Advanced with twist grip	(36 03 075)				
24"	Beginner with twist grip	(36 04 072)	6 km/h	mild / 6 km/h	medium / 8 km/h	strong / 12 km/h
	Beginner with button	(36 04 073)				
	Advanced with twist grip	(36 04 074)	6 km/h	mild / 10 km/h	medium / 15 km/h	strong / 20 km/h
	Advanced with twist grip	(36 04 075)				
26"	Beginner with twist grip	(36 05 072)	6 km/h	mild / 6 km/h	medium / 8 km/h	strong / 12 km/h
	Beginner with button	(36 05 073)				
	Advanced with twist grip	(36 05 074)	6 km/h	mild / 10 km/h	medium / 15 km/h	strong / 20 km/h
	Advanced with twist grip	(36 05 075)				



2. Product description.

2.8 Drive possibilities

The **momo therapy bicycle**. can be equipped with the following drives:

Rigid sprocket

During active pedaling, the rigid sprocket supports the user in overcoming the "dead"/apex point of the crank. A passive user is assisted in pedaling constantly; in addition it permits travel forwards or backwards.



Freewheel brake hub (with backpedal brake)

The freewheel brake hub allows the user to stop and start pedaling as they wish. They can brake by pedaling backwards.

Extra-light transmission

An extra-light transmission can be selected for all the above-mentioned types of drive. Here a larger pinion on the rear axle permits improved force transmission and thus makes it easier for the user to start pedaling.

3 or 7-gear freewheel brake hub (with backpedal brake)

Using the 3 or 7-gear freewheel brake hub, the user can stop and start pedaling at will. They can brake by pedaling backwards. The user-friendly 3 or 7-gear hub gear system permits switching of gears when the tricycle is at a standstill. Switching takes place using the twist grip shifter on the handlebars.

3 or 7-gear freewheel brake hub with reverse option

Using the 3 or 7-gear freewheel brake hub, the user can stop and start pedaling at will. In addition, they can ride backwards. The user-friendly 3 or 7-gear hub gear system permits switching of gears when the tricycle is at a standstill. Switching takes place using the twist grip shifter on the handlebars.



7-gear freewheel brake with wheel brake (without backpedal brake)

In the 7-gear freewheel hub with wheel brake forward, the movement can be interrupted as desired and be converted in free backwards pedaling in further forward drive. The user-friendly 7-gear hub gear system is suitable for any terrain and also permits switching of gears when the bicycle is at a standstill. Switching takes place using the twist grip shifter on the handlebars.

2. Product description.

2.9 Adjusting the stabilisers

The stabilisers are equipped with spring elements that can be adjusted from soft to hard. To do this, loosen the screw **(A)** and move the spring element in the slot. Moving towards the stabiliser increases the spring hardness. By loosening the two screws **(B)**, the stabilisers can be adjusted so that they are off the floor. Tighten the screws again after all adjustments. In addition, spring travel can be limited by means of an adjustable end stop. To this end, loosen the counter nut **(C)** below the support. Spring travel can now be adjusted by screwing in or out the screw **(D)**, re-tighten the counter nut afterwards.

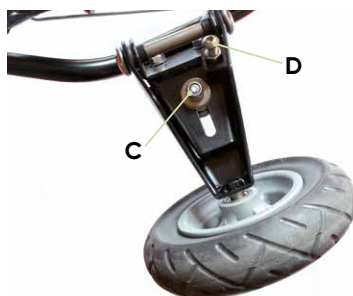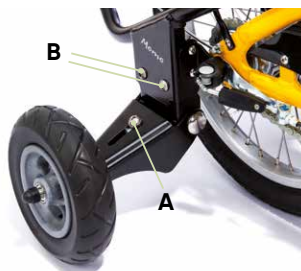

2.10 The initial riding attempts

Please remember that, prior to setting off on your own, you must first practice with an accompanying person in order to learn how to deal with riding around curves on the therapy bicycle, as this is where the risk of accidents is greatest. Always ride around curves as slowly as possible. Please also keep in mind that the rear is the widest point of the therapy bicycle. Therefore practise with corresponding obstacles in order to better estimate the width of the therapy bicycle.



3. Settings.

Settings and adjustments to the product or accessories may only be made by people who have been given the necessary instructions by a medical product advisor. Please ensure that none of the user's extremities are in the respective area when making adjustments of any kind to minimise the risk of injury. All adjustments can be made with standard tools (e.g. Allen key, screwdriver or spanner).

3.1 Presettings


The **momo therapy bicycle**. is delivered completely assembled. Prior to initial use, however, the following pre-settings must be made.

3.1.1 Handlebar adjustment

You can find our scope of delivery for handlebars on page 7 of these Instructions for use.

Height of handlebars

To adjust the height of the handlebars, remove the protective cap from the hexagon socket (A), loosen the hexagon socket (A) and adjust the stem (B) to the required height. By tapping the head of the hexagon socket (E) lightly with a hammer, the stem in the fork steerer will loosen. Then re-tighten the hexagon socket.

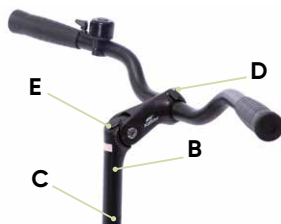

Ensure that the marking for the minimum insertion depth (C) remains on the stem in the fork steerer and therefore cannot be seen.

Handlebar adjustment

To adjust the handlebar position, loosen the clamping screws (D), bring the handlebars into the required position and then firmly re-tighten the clamping screws (D).

To change the tilt angle on the stem, please loosen the clamping screw (F). The steerer factory setting is 20°. In this way you can change the distance between the saddle and the handlebars, and adjust the required handle height. Then firmly re-tighten the screws again.



After each adjustment, please retighten all screw connections!



After adjustment of the handlebars, there must still be no tension placed on the brake cables. If necessary, extend the cables!

3. Settings.

3.1.2 Saddle adjustment

You can find our selection of saddle forms on page 9 of these Instructions for use.

Saddle height

The height of the saddle can be adjusted on the frame saddle tube (A) by pulling the saddle post in or out. In order to adjust the height, please loosen the clamp (B) and bring the saddle into the required height. Align the saddle and tighten the clamp (B) so tightly that the saddle support no longer twists. The saddle height is to be re-set based on the inseam length. Here any contractures of the knees must be taken into consideration. The saddle height should be inspected when the user is sitting on the saddle. The leg extension should not total 0°. In the upper position of the pedal, the knee flexion should not be more than 90°. Should it not be possible to achieved this setting through the saddle height, it is possible to undertake further settings through crank shortening (see point 4.9 – 4.11).




Adjusting the saddle horizontally

Every saddle (except for the unicycle saddle) can be minimally adjusted horizontally. To do this, loosen the nuts (D) with a size 13 wrench and push the calliper block (C) forwards or backwards on the seat stays.



Adjusting the saddle with t-saddle post horizontally

In order to check the horizontal saddle position, turn the pedal crank horizontally forwards and put the foot (while the rider is sitting on the saddle) onto the pedal. Once the lower leg is vertical, the saddle position is correct. You can also loosen the two nuts (D) under the saddle using a size 13 wrench, push the saddle to the rear or forwards and re-tighten the nuts.

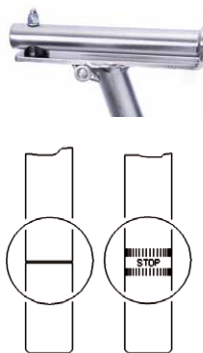

During adjustment, the saddle post may not be pulled out over the respective marking, as in this case sufficient clamping cannot be guaranteed. The markings are in part designed differently.



After each adjustment, please retighten all screw connections!




3. Settings.

3.2 Brakes

For the **momo therapy bicycle**, there are different brakes available, depending on the individual requirements of the user.

3.2.1 Parking brake

The parking brake supports the user when getting on and off and also secures the therapy bicycle against unintentional rolling away. To activate the parking brake, press the lever (A) on the handlebar stem (B) downwards. To release the parking brake, push the lever up again.

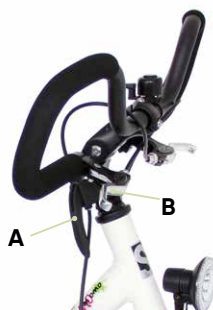

Always ensure that the brake or the brake blocks function correctly and have been adjusted (see below).

Adjustment of the parking brake or brake blocks

Ensure that the braking function immediately takes effect on actuation of the parking brake lever, taking the necessary backlash into account.

Due to "settling" of the Bowden cables and the natural wear on the brake blocks, it is necessary to reset the brake or replace the brake blocks after a certain amount of time.

Each gap between the rim and the brake lining should not be larger than 1.5 mm. To do this, loosen the clamping screw (C), press the brake blocks together by hand, pull the Bowden cable taut and then re-tighten the clamping screw. If the brake blocks are not accurately aligned with the rim edge (D), you must readjust them accordingly. In order to change the alignment of the brake blocks, loosen the screws (E). Leave this work to your specialist dealer in case of any uncertainty.



After each adjustment of the cantilever (V) brake, carry out a brake test. The brake blocks may only be replaced by ones in an identical design. Observe the manufacture name or marking and the type designation. New brake blocks do not achieve the required braking effect until they have been used multiple times.

3. Settings.

3.2.2 Round handlebars with brake lever ring

Braking with round handlebars with brake lever ring is done by gently pushing down the ring (A) and is therefore suitable for users with limited strength in arms and hands who require the round handlebars as support point at the same time. The brake lever ring is coupled with the rim brake.



3.2.3 Round handlebars with brake function

The braking function is triggered by simply pressing the round handlebars (B) downwards. The round handlebars with braking function is suitable for users with limited strength in arms and hands.



3.2.4 Drum brake in front wheel

The brake lever for actuation of the drum brake is located on the right-hand side of the handlebars. In drive mode, always use only the front wheel drum brake in connection with the backpedal brake.



The braking effect can be reduced if the brakes are used frequently. Therefore ensure to regularly re-adjust the Bowden cable of the brake when you notice a loss in braking power (see Point 3.2.1)



3.2.5 Backpedal brake

The backpedal brake is actuated by pedaling backwards. The drive options of the freewheel brake hub and the 3 or 7-gear freewheel brake hub feature a backpedal brake.



The backpedal brake is only functional when the chain sits correctly! If the chain jumps off, it is not possible to brake using the backpedal brake!



The rear wheel may block in case of strong braking procedures. Risk of crashes!



During long downhill runs, it is essential that you use the front and rear brakes in order to avoid overheating of the rear hub brake. This may lead to brake malfunctions!



3. Settings.

3.3 Tyres and hoses

The tyres of the therapy bicycle must always have enough air pressure, as otherwise the tyres could puncture and the rims get damaged or the driving behaviour is negatively affected. The optimal tyre pressure is marked on the tyre casing (A). If the tyre tread only depresses slightly on being pressed forcefully with the thumbs, the tyre pressure is correct. For exact values, use a pressure gauge!



Check all tyres regularly and replace them immediately in case of damage or wear!

3.4 Chain tension

Due to the elongation of the chain caused by use, the chain tension must be checked regularly. Check the chain tension by testing whether the chain of the therapy bicycle can be pushed max. 10 – 15 mm up and down. To tension the chain, remove the stabiliser frame (B), loosen the axle nuts (C) using a 15mm open-end wrench and pull the rear wheel (D) towards the back uniformly on both sides. If the therapy bicycle is equipped with a gear hub (3 or 7 gears and idle brake hub), you must also disassemble the brake anchor and the click box of the gears. Leave this work to your specialist dealer in case of any uncertainty!



After each adjustment, please retighten all screw connections!



An incorrectly-tensioned chain can lead to increased wear!

3.5 Light system / dynamo

Side dynamo: The lighting system is activated by pressing the switch (E) of the side dynamo. The dynamo roller then lies automatically against the tyre flank.



Hub dynamo: The hub dynamo is automatically integrated into the wheel and is switched on via the switch (F) on the headlight.





4. Accessories.

4.1 Dynamic back and pelvic guide pelotte pads

All back and pelvic guide pelotte pads can only be used in connection with a holding bracket (see **Point 4.7**). For depth adjustment of the pelotte pad holding bracket, please loosen the screws (**A**) on the right and left-hand sides of the support under the saddle, and bring the holding bracket into the required position.

The angular adjustment of the holding bracket takes place after loosening the screw (**B**) on the support. The pelotte pads are adjusted in height after loosening the screw (**C**) on the respective support.

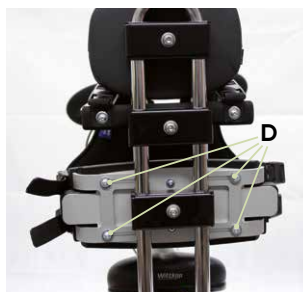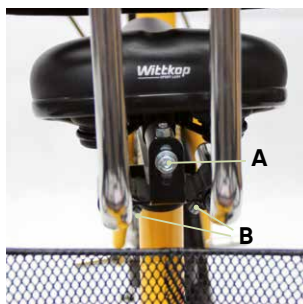

 **Use only original accessories as otherwise the warranty will become void.**

4.1.1 Width adjustable back and pelvic guide pelotte pads

To adjust the height, handle and depth of the back and pelvic guide pelotte pads, see **Point 4.1**. The back and pelvic pelotte pads can be adjusted in width. To do this, loosen the screws (**D**) on the rear of the back or pelvic guide pelotte pads and bring them into the required position.

4.2 Headrest

The headrest can only be used in connection with a holding bracket (see **Point 4.7**) and is adjustable in height. In order to adjust the height, loosen the screw (**E**) and bring the headrest into the required position.




 **After each adjustment, please retighten all screw connections!**



4. Accessories.

4.3 Push bar


The push bar is height adjustable and equipped with an integrated antirotation lock. In addition, it can be dismantled. In order to adjust the height, please loosen the clamp (A) and bring the push handle to the required height. In order to remove the entire push bar, please loosen the clamp (B). Please ensure on insertion that the push bar is inserted up to its limit into the push handle holder, and cannot be rotated.

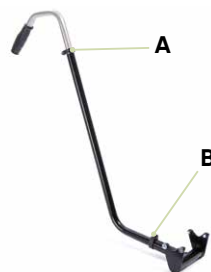

Use the push bar only to guide the therapy bicycle! The push bar is NOT suited for repositioning the therapy bicycle or lifting or tilting!

4.4 Parking brake for accompanying escorts

The parking brake for accompanying persons (can only be used in connection with the push rod) secures the therapy bicycle against unintentional rolling away. The function of the lever is similar to a normal brake lever. For this reason, the parking brake can also be used as a normal brake. If however the brake should be locked in parking position, the claw (D) must be pressed into the ratchet with the brake lever (C) pulled. Loosen the locking device by pulling the brake lever (C).

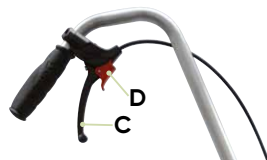

Always ensure that the brake and the brake pads work and that they have been adjusted (see Point 3.2.1).

4.5 Steering for accompanying persons

Steering for accompanying person (by means of steering linkage, incl. handlebar lock limiter) allows the accompanying person to help determine the direction of travel of the therapy bicycle by shifting the front wheel without interrupting the user's pedaling movements.

4. Accessories.

4.6 Handlebar lock limiter

The handlebar lock limiter can be adjusted to determine the direction of travel. In order to adjust the handlebar lock limiter, please loosen all grub screws (A) and bring the limiter (B) into the required position. You can determine the direction by moving both limiters (B) towards the handlebar stop (C) and tightening them.




If possible please leave the settings on the handlebar lock limiter as set at the factory!

4.7 Holding bracket with mount

The holding bracket with mount permits the mounting of head, torso or pelvic supports that provide extra stability to the therapy cyclists. Included in the scope of delivery is the black bracket adapter, which is mounted onto the T-saddle post. Then the holding bracket is pushed into the holding socket (D) and can be adjusted in depth. To do this, simply adjust the required position and then tighten the screws (E).



The use of the holding bracket is exclusively possible in combination with the T-saddle post!

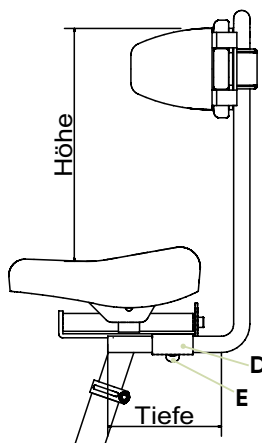

4. Accessories.

4.8 Universal adapter

The universal adapter is mounted to the holding bracket and is used to mount the belt sets available for the therapy bicycle. The universal adapter is adjustable in height. To do this, simply loosen the screw (A) at the rear on the adapter and bring the universal adapter into the required position.



4.9 Crank shortener (continuously adjustable)

The adjustable crank shorteners are to be mounted onto the cranks with clamps. Please note here that the crank shortener marked "R" is to be mounted on the right-hand side and the crank shortener marked "L" is to be mounted on the left-hand side. The adjustment of the crank shortener is to be undertaken so that the amplitude of the pedal depicts the flexibility of the knee joint. In the lower pedal position, the maximum extension to be achieved, and in the upper position, the maximum flexion of the knee is to be achieved. This adjustment is to be undertaken in interaction with the saddle height. In order to adjust the continuously adjustable cable shortener, loosen the screws (B) and bring them into the required position.



4.10 Crank shortener

The crank shortener must be mounted onto the cranks using the screw included in delivery and the clamp, and shortens the crank by 2.5 or 5 cm. Please note here that the crank shortener marked "R" is to be mounted on the right-hand side and the crank shortener marked "L" is to be mounted on the left-hand side. The pedal is to be mounted in the required position in the crank shortener.

The position of the pedals must be conducted so that the amplitude of the pedal depicts the flexibility of the knee joint. In the lower pedal position, the maximum extension to be achieved, and in the upper position, the maximum flexion of the knee is to be achieved. This adjustment is to be undertaken in interaction with the saddle height. In order to adjust the crank shortener, loosen the pedals with a 15mm open-end wrench and place it into the opening (C).



4. Accessories.

4.11 Special crank for knee contracture

The special crank for knee contractures is suitable for 20" - 26" tricycles, and can either be mounted on the left or right-hand side.



5. Foot positioner.

5.1 Exercise bike pedals

The exercise bike pedals are characterised by their integrated balancing weight, whereby the tread surface automatically balances itself horizontally. This permits the rider to climb on independently. The strap (A) is adjustable in length, and at the same time provides easy side guidance. To adjust the length of the strap (A) pull on the lower end in order to loosen the strap (A) from the plug (B). Now adjust the strap (A) to the required length.



5.2 Foot positioning pedals



The foot positioning pedals are characterised through the integrated balancing weight, whereby the treads are automatically horizontally balanced. This permits the rider to step onto them independently. The strap (C) is adjustable in length, and at the same time provides easy side guidance. The cage mounted at the front of the pedals (D) prevents the toes from slipping through at the front. In order to adjust the strap (C) in length, pull the strap (C) back out of the strap guide (E). Now the strap (C) can be shortened through pulling and extended through pressing under the lock. To fix the adjustment, guide the strap (C) back through the strap guide (E).



5. Foot positioner.

5.3 Foot pans


The foot pans provides side guidance and thus prevents internal or external rotations of the foot. To guarantee this function, the foot pan is adjustable in width, which can be conducted by loosening the three screws (A) and shifting the side part (B) in the slotted hole. The foot pans are mounted centrally on the pedals in the factory. In order to shift the pressure point under the foot, it must be moved in three positions. To this end, remove the screws with nuts (C) below the counterplate and move it into the desired position. Subsequently put the counterplate back and tighten again the screws with nuts (C). The rotation can be adjusted by loosening the screws with nuts (C) and turning the foot pans on the pedal. In order to secure the user in the foot pan, the foot pans are equipped with straps. These can be closed using the magnetic locks. If you guide the strap end (D) in direction of the closure, it will engage automatically. Press the button (E) to open the straps. For the length adjustment, the hook and loop closure of the strap (F) can be opened on both sides and adjusted in length. To close the strap in the forefoot area, guide it through the opening (G) and close the strap with hook and loop closure.

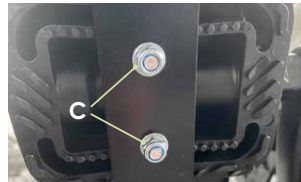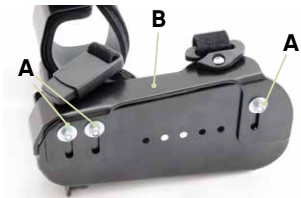

5. Foot positioner.

5.4 Foot pans with leg guidance

See **Point 5.3** for the function and settings of foot pans. The leg guidance also stabilises the foot joint and reduces internal rotation of the leg. The leg guidance can be adjusted by loosening the nuts (**A**) and shifting them in the slotted hole. The height adjustment should be selected so that the calf clamp lies against the vertex of the calf. The depth of the calf clamp can be adjusted by loosening the nuts (**B**) on the inside of the leg guidance.

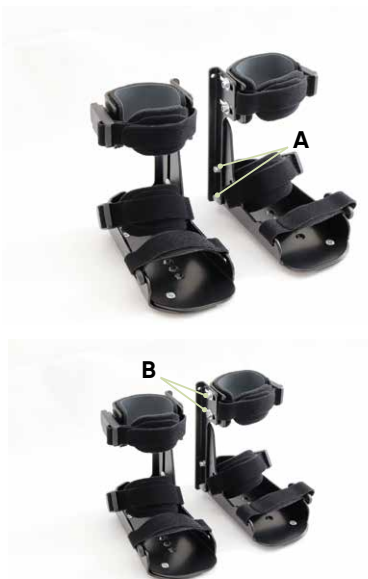

5.5 Foot pans with dynamic leg guidance

For the function and adjustment of the foot pans with leg guidance, see **Point 5.4**. For the foot pans with dynamic leg guidance, the adjustment options are installed only on the rear foot part and not on the side like for the foot pans with leg guidance.

The dynamic leg guidance also permits defined rotation of the leg and thus prevents excessive abduction, in particular of short legs. At the same time, the stabilisation of the foot joint is retained. To adjust the degree of movement of the leg guidance, loosen the cover (**C**) and the nut below it, and screw the elastomer in or out accordingly. Check the leeway of the leg guidance.



6. Strap systems.

6.1 Chest strap

The chest strap is attached to the width adjustable back pelotte pad, and, if required, ensures secure positioning of the user. The strap is mounted with the aid of the plug lock to the back pelotte pad and threaded through the strap guidance on the click buckle. Then the click buckle is pressed down to fix.



6.2 Positioning vest

The positioning vest is mounted in the lower area of the width adjustable back pelotte pad. Screw the strap ends of the positioning vest onto the universal adapter and tighten the screws (A). The strap of the positioning vest is mounted onto the back pelotte pad with the aid of the plug lock, and threaded through the strap guidance on the click buckle. Then the click buckle is pressed down to fix.



6. Strap systems.

6.3 Groin harness, T-shaped

The T-shaped groin harness is mounted onto the width adjustable pelvic pelotte pad. The strap is mounted onto the pelvic pelotte pad with the aid of the plug lock and threaded through the strap guidance on the click buckle. Then the click buckle is pressed down to fix.



6.4 4-point pelvic harness

The 4-point pelvic harness is mounted onto the width adjustable pelvic pelotte pad. The strap is mounted onto the pelvic pelotte pad with the aid of the plug lock and threaded through the strap guidance on the click buckle. Then the click buckle is pressed down to fix.



6.5 Hand positioning aid

The hand positioning aid provides additional security for the user by keeping their hands safely on the handlebars. To do this, the user simply inserts their hand into the hand positioning aid and closes the hook and eye fasteners.



7. Electric drives.

7.1 E-drive

The **momo therapy bicycle.** can optionally be equipped with an E-drive (Heinzmann Direct Power motor (36V, 250W, impulse torque 60Nm).

Switching the E-drive on and off

To switch the e-drive on and off, press and hold the "MODE" button (A).

Switching the lighting on and off

To turn the lights on and off, press and hold the button (B).

Selecting/changing assistance levels

To select or change the assistance level (Eco, Standard or Power), briefly press the buttons (B or C). The set level can also be read from the bar (D) on the display.

Activate start-up assistance by button (if integrated)


To activate the start-up assistance press and hold the button (C).

Activate start-up assistance via twist grip (if integrated)

To activate the start-up assistance, turn the twist grip (E) towards the body.

Brake handle

When the brake handle (F) is actuated, the e-drive is switched off and reactivated as soon as the brake handle (F) is released.

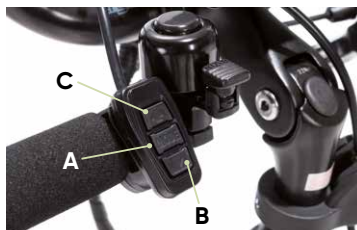

7. Electric drives.

Battery

The battery can be locked and removed for charging. To charge, plug the charging cable (A) into the rear of the opening provided. You can read the progress of loading at any time via the charge level indicator (B). The charging time totals approx. 6 hours.



Battery service life

The manufacturer of the E-drive, HEINZMANN, guarantees a minimum of 600 charging cycles for the battery. Appropriate handling / storage and initial loading (see HEINZMANN user manual) increases its service life.

Range

The mileage of the **momo therapy bicycle** with E-drive depends on different factors: track layout (gradients), load/rider weight, selected assistance level, riding style. HEINZMANN achieves a range of 40 to 80 km per battery charge.



Please also note the HEINZMANN instructions for use.

7.2 Light system

The light system of the **momo therapy bicycle** with E-drive is supplied via the Li-ion high-performance battery pack (11Ah).



8. Cleaning and servicing.

8.1 Cleaning and disinfecting

8.1.1 Cleaning

To ensure functional safety and the appearance of the bicycle, it must be regularly cleaned and cared for. Please observe the following guidelines:

- Never allow dirt to dry onto the bicycle, but always remove it with water and a soft cloth or sponge. Otherwise you may damage the bearings, the paintwork or the decorations.
- Do not use aggressive cleaning agents. Use a mild detergent solution to clean.
- Repair paintwork damage immediately.
- The components of the drive system can be cleaned with a soft cloth moistened with commercially available cleaning products or soapy water, but not wet.

Please also pay attention to our general cleaning and hygiene advice. This can be found at www.schuchmann.de/mediathek.



Danger due to rotating parts

The motor can set itself in motion unexpectedly, if the drive system remains switched on when carrying out cleaning work on the **momo therapy bicycle** with E-drive.

- Remove the E-drive battery before cleaning.



Never use high-pressure cleaning equipment

The use of a steam jet, high-pressure cleaning equipment or a water jet for cleaning is not permitted. The ingress of water into the control box, motor or electrical connectors can destroy the equipment.

8.1.2 Disinfection

Various products can be used for surface disinfection of metal and plastic parts.

Liquid disinfectants are available as ready-to-use solutions that are sprayed on and evenly applied with a soft cloth. Alternatively, wipes pre-soaked with disinfectant can be used to wipe the products over the entire surface. In both cases, care must be taken to ensure complete wetting. Disinfection in fully automatic disinfection systems is also possible and recommended. The exposure times may vary and can be found in the manufacturer's instructions for the products used.

8. Cleaning and servicing.

8.1.3 Chain maintenance

Drive chains must be regularly cared for. This is in particular the case after riding in rain. The chain must be lubricated with a commercially available chain oil. Due to the expansion of the chain which is a natural result of use, regular inspection of the chain tension is required (see **Point 3.4**).

8.2 Servicing / controls

Please perform a daily visual inspection and regularly check the tricycle for cracks, fractures, missing parts, malfunctions, and secure screw connections. In case of a defect or malfunction, please contact the specialist dealer who supplied you with the product (see **Point 11.4**).

8.3 Maintenance

For reasons of user safety and to maintain product liability, maintenance needs to be carried out every 1000 km, however at least once a year, by a specialist dealer (see **Point 11.4**). The maintenance work carried out must be documented in the maintenance plan (see **Point 8.3.2**). The electric drive (incl. battery pack) does not require maintenance if used properly and carefully.

8.3.1 Maintenance specifications

- Check the chain and chain tension, adjust if necessary, clean and oil.
- Check rear wheel track, adjust if necessary.
- Check bottom bracket and lubricate if necessary.
- Lubricate pedal bearing, check bearing clearance and if necessary adjust or replace.
- Check the hub gear and adjust if necessary.
- Check the brake system for function, adjust if necessary. If the braking effect is poor, check the condition of the hand lever, cable, brake lever and brake lining, readjust and replace if necessary.
- Lubricate joints and bearing points.
- Replace kinked or clamped cables.
- Check rims for side and top impacts.
- Check the spoke tension and adjust if necessary.
- Check the tyre profile.
- Check the lighting and signal system.
- Check rear hub and lubricate if necessary.
- Check frame and fork for damage and replace if necessary.
- Have a current leakage measurement of the E-drive carried out annually.
- Check the fixation of all cables and parts.
- Check the function of the entire electrical system.
- Check the operational safety of the battery.
- Retighten screw connections using the specified torque values indicated on the respective components, if applicable.



8. Cleaning and servicing.

Danger of malfunction in case of incorrect maintenance

Improper maintenance of the electric drive can cause damage to essential components. This might cause a fall.

- Maintenance work may only be performed by your specialist dealer (see **Point 11.4**).

Danger due to unforeseen motor activity

The motor can set itself in motion unexpectedly, if the drive system remains switched on when working on the bicycle.

- Before each intervention, remove the battery pack from the bicycle with E-drive.
- When restarting the bicycle after maintenance, assembly or repair, position the bicycle so that the drive wheel can rotate freely. Only then should the battery be inserted and the proper functioning of the drive checked.

8.3.2 Maintenance plan

Maintenance specifications of the manufacturer (see **Point 8.3.1**) were carried out:

Date	Company	Name	signature



Any defects or damage found must be repaired by the specialist dealer or the manufacturer before reuse.

8. Cleaning and servicing.

8.4 Repairs

Bicycle repairs that are not performed by your specialist dealer are at the user's own risk and discretion.




Caution

When reinstalling the drive, e.g. after a tyre change on the front wheel, the wheel nuts must be tightened again with a tightening torque of $45 \text{ Nm} \pm 5 \text{ Nm}$!

The torque support must be fully encircled by the dropout. The reserve in the mounting groove for the wheel axle in the dropout must be at least 5 mm!

The shoulder (**A**) of the torque support must point to the open side of the dropout!

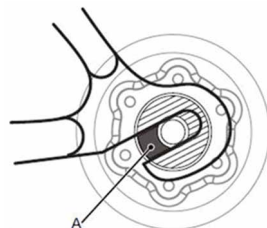

8.5 Controls

Controls to be performed by the user of the bicycle if necessary:

- Check the chain and chain tension, adjust if necessary, clean and oil.
- Check the chain for wear, oil and replace if necessary.
- Check bottom bracket mounting and repair if necessary.
- Check pedals for clearance.
- Gear system – check settings.
- Check the handlebar and handlebar stem for damage and replace if necessary.
- Check the brake system for function and adjust if necessary.
- Check the tyre pressure and profile.
- Check the lighting and signal system.

8.6 Storage

Storage in winter

Prior to storing the bicycle in winter in a dry room with a constant temperature, clean it (see **Point 8.1**) and ensure that the tyre air pressure is sufficient (see **Point 4.3**).

Inspection in spring

Prior to using the bicycle again in spring, ensure that the tyre air pressure is sufficient and that there is no damage to the bicycle.



8. Cleaning and servicing.

8.7 Spare parts

Only use accessories and spare parts made by Schuchmann, otherwise you will endanger the user and the warranty becomes void.

Please contact the supplying specialist dealer to order spare parts stating the serial number of the bicycle (see **Point 11.4**). Necessary spare parts and accessories must only be installed by trained personnel.

8.8 Duration of use and re-use

The expected duration of use of our product, dependent on the usage intensity and amount of re-use, totals up to "8" years, if the usage takes place in accordance with the information in these instructions for use. It may be possible to use the product over and above this time period if it is in a safe condition. The expected duration of use does not refer to wear parts, such as for example covers, wheels, batteries.....The maintenance and evaluation of the condition, and if applicable the potential for re-use, must be decided by the specialist dealer.

The bicycle is suited for reuse. Prior to forwarding, please follow the cleaning and disinfection instructions stated in **Point 8.1**. Accompanying documents such as these Instructions for use are part of the product and must be passed on to the new user.



Should a serious incident occur during the service life of the product despite being used as intended, this must be reported immediately to the manufacturer and the competent authority.

8. Cleaning and servicing.

8.9 Error messages and troubleshooting

If errors occur during operation of the drive system, “**Error**” is indicated on the display. At the same time, instead of the current display of the speed, an error number appears.

The following table provides information about the possible error sources associated with these numbers and how the malfunction can be eliminated.

1	ERROR_HARDWARE_BRAKE
Error	Overcurrent switch-off
Measure	Remove the battery and check the contacts. If the contacts are OK, reinsert the battery and restart the system. Please contact your specialist dealer (see Point 11.4) if the contacts on the battery pack are visibly damaged or the error continues to occur.

2	ERROR_HALL
Error	Motor signal faulty
Measure	Check the motor signal cable and restart the system. Please contact your specialist dealer (see Point 11.4) if the motor signal cable are visibly damaged or the error continues to occur.

3 & 4	ERROR_GRIP_OFFSET
Error	twist grip not in the home position at system startup
Measure	Make sure that the twist grip is in the home position at system startup. If the error still occurs, the twist grip is probably defective. In this case, please contact your specialist dealer (see Point 11.4).

5	ERROR_TORQUE_OFFSET
Error	Torque sensor incorrectly installed or not load-free at system startup
Measure	Do not apply pressure to the pedals on system startup. If the error still occurs, the torque sensor in the bottom bracket is probably defective. In this case, please contact your specialist dealer (see Point 11.4).

6	ERROR_I_OFFSET
Error	Current measurement error, contacts on the battery may be damaged
Measure	Remove the battery and check the contacts. If the contacts are OK, reinsert the battery and restart the system. Please contact your specialist dealer (see Point 11.4) if the contacts on the battery pack are visibly damaged or the error continues to occur.



8. Cleaning and servicing.

7	ERROR_FAST_OVER_VOLTAGE
Error	Brief overvoltage in the intermediate circuit, possibly due to excessive wheel speed or damaged contacts on the battery.
Measure	Remove the battery and check the contacts. If the contacts are OK, reinsert the battery and restart the system. Please contact your specialist dealer (see Point 11.4) if the contacts on the battery pack are visibly damaged or the error continues to occur.
8	ERROR_SLOW_OVER_VOLTAGE
Error	Permanent overvoltage in the intermediate circuit.
Measure	Restart the system. Please contact your specialist dealer (see Point 11.4) if the error continues to occur.
9	ERROR_FAST_UNDER_VOLTAGE
Error	Brief undervoltage in the intermediate circuit, battery possibly took weak or contacts damaged.
Measure	Remove the battery and check the contacts. Reinsert the rechargeable battery and restart the system. If the error still occurs, charge the battery pack, reinsert it and restart the system. Please contact your specialist dealer (see Point 11.4) if the contacts on the battery pack are visibly damaged or the error continues to occur.
10	ERROR_SLOW_UNDER_VOLTAGE
Error	Permanent undervoltage in the intermediate circuit, rechargeable battery possibly drained.
Measure	Charge the battery, reinsert and restart the system. Please contact your specialist dealer (see Point 11.4) if the error continues to occur.
11	ERROR_OVER_TEMP_MOTOR
Error	Motor overheated
Measure	Leave the motor to cool down. Restart the system. Please contact your specialist dealer (see Point 11.4) if the error continues to occur after approx. 20 minutes.
12	ERROR_OVER_TEMP_CONTROLLER
Error	Motor control overheated
Measure	Leave the motor control to cool down. Restart the system. Please contact your specialist dealer (see Point 11.4) if the error continues to occur after approx. 20 minutes.
13	ERROR_PARAMETER
Error	General parameter error
Measure	Restart the system. Please contact your specialist dealer (see Point 11.4) if the error continues to occur.

8. Cleaning and servicing.

14	ERROR_UNDER_TEMP_MOTOR
Error	Lower limit temperature for motor operation reached
Measure	Store the system at room temperature (approx. 20°C). Please contact your specialist dealer (see Point 11.4) if the error continues to occur after approx. 1 hour.

15	ERROR_EEPROM
Error	Error in the system accumulator
Measure	Restart the system. Please contact your specialist dealer (see Point 11.4) if the error continues to occur.

16	ERROR_PARAMETER_PROPERTY
Error	Error in parameter management
Measure	Restart the system. Please contact your specialist dealer (see Point 11.4) if the error continues to occur.

17	ERROR_AKKU_TEMP
Error	Battery either overheated or under-cooled
Measure	Allow the overheated battery to cool down or store under-cooled at room temperature (about 20°C). Restart the system. Please contact your specialist dealer (see Point 11.4) if the error continues to occur after approx. 1 hour.

18	ERROR_UNDEFINED_BIKE_CONSTELLATION
Error	Configuration error
Measure	Restart the system. Please contact your specialist dealer (see Point 11.4) if the error continues to occur.

19	ERROR_BMS
Error	Rechargeable battery error
Measure	Remove the rechargeable battery and recharge. Reinsert the rechargeable battery and restart the system. Please contact your specialist dealer (see Point 11.4) if the error continues to occur.

20	ERROR_BMS_VERSION
Error	Rechargeable battery error
Measure	Restart the system. Please contact your specialist dealer (see Point 11.4) if the error continues to occur.



8. Cleaning and servicing.


21	ERROR_BMS_VERSION
Error	Loose contact or other defect on the torque sensor
Measure	Restart the system. Please contact your specialist dealer (see Point 11.4) if the error continues to occur.

22-30	ERROR_BMS_FAULTY_xxx
Error	Rechargeable battery error
Measure	Please contact your specialist dealer (see Point 11.4).

50	ERROR_DISPLAY_COMMUNICATION
Error	Communication between the display and control faulty
Measure	Check the display cable plug. Examine the display for visible damages. Restart the system. Please contact your specialist dealer (see Point 11.4) if damage can be identified on the plug or display or the error continues to occur.

9. Technical data.

9.1 Dimensions

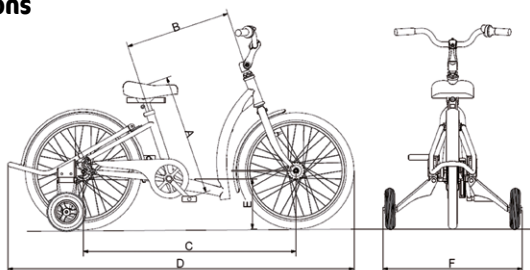

		12"	16"	20"	24"	26"
A*	Pedal to top edge of saddle	30 - 38 cm	42 - 57.5 cm	50 - 64.5 cm	59 - 76.5 cm	71.5 - 87.5 cm
A**	Pedal to top edge of saddle	33 - 41 cm	45 - 60 cm	52 - 68 cm	61 - 84 cm	73 - 95 cm
B	Saddle support tube to front tube	33 cm	39 cm	45 cm	53 cm	58 cm
C	Wheelbase	64 cm	76 cm	89 cm	104 cm	112 cm
D	Full length	99 cm	122 cm	148 cm	168 cm	178 cm
E	Access point height	16 cm	18 cm	20 cm	22 cm	23 cm
F	Full width	43 cm	52 cm	62 cm	71 cm	76 cm
	Tare weight	10.5 kg	13 kg	15 kg	18 kg	20 kg
	max. load	35 kg	50 kg	65 kg	100 kg	100 kg
* Dimensions with standard saddle post / ** Dimensions with T-saddle post						

9.2 Tyre pressure

The minimum and maximum tyre pressure is indicated on the casing (**A**).



9.3 Torque information

Pedal crank	40 Nm
Front wheel	20 Nm - 22 Nm
Heinzmann wheel hub motor	35 Nm
Rear wheels not powered	25 Nm - 30 Nm



9. Technical data.

9.4 E-drive

system	
Protection class	3

Control	
Operating voltage	36 V
Max. current	depending on design & operation: 10 - 33 A

Motor	
Type	PRA 180-25
Motor nominal voltage	22.8 VAC
Output	250 W Pedelec (DIN EN 60034-1)
Speed during travel on the flat	depending on design & wheel diameter approx. 60-330 1/min
Torque	11.4 Nm
Impulse torque	up to 60 Nm
Overtemperature protection	Type KTY84-130
Overall motor diameter	Ø 220 mm
Weight	4.5 kg
Protection class	IP54
Consumption	0.5 – 1 kWh/100km

Battery		
Type		Lithium ion battery
Nominal voltage		36 V
Capacity		11 Ah, 400 Wh
Charging time		approx. 5.5 h (at charge current 2 A)
Temperature ranges	Operation	(-)10 - 45°C
	Charging	10 - 35°C
	Storage	(-)10 - 45°C

9. Technical data.

9.5 Lighting system

Front lighting	
Type	AXA Pico 30-E 6V-42V
Luminous intensity	30 lx
See	50 m
Be seen	3000 m
Lateral visibility	Yes
Including reflector	Yes
On/off switch	Yes
Automatic light/dark sensor	No
Parking light	No
Daytime running light	No
Suitable for E-bike	Yes, 6-42 V
Primary colour	black
Product weight	70 g
Technology	LED

Rear lighting	
Type	Busch und Müller Seculite Plus
Parking light	Yes
Registration light	StVZO
Assembly	Mudguard
Technology	LED
Integrated reflector	Yes

10. Guarantee.


The two-year statutory guarantee period shall apply for all products. This begins with the delivery or handover of the goods. Should a verifiable material or manufacturing fault occur within this time period, we shall, after carriage paid return to us, view the indicated damage and, if applicable, either repair or deliver a new product at our discretion.



11. Identification.

11.1 Serial number / date of manufacture

The serial number, the date of manufacture and other information can be found on the type plate, which is located on all of our products (A).

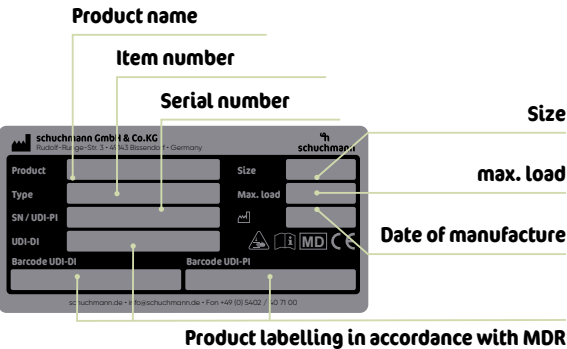

11.2 Product version

The **momo therapy bicycle**. is available in five sizes (12" – 26") and can be supplemented through a diverse range of accessories (see **Point 4**).

11.3 Issue of the document

Instructions for use **momo therapy bicycle**. – Change status M; Issue 05.2025

11.4 Name and address of the manufacturer, specialist dealer supplying the product

This product was manufactured by:



Schuchmann GmbH & Co. KG
Rudolf-Runge-Str. 3 · 49143 Bissendorf · Germany
Phone +49 (0) 5402 / 40 71 00 · Fax +49 (0) 5402 / 40 71 109
info@schuchmann.de · www.schuchmann.de

This product has been delivered by the following specialist dealer:





schuchmann.de