

General information.



Look, the stain has disappeared!



„Now let's wipe down the table - and **tim.** will soon be sparkling clean again!" Not just **tim.**, but all our aids can be cleaned easily and kept free from dirt. On the following pages, we'll show you exactly how they should be cleaned and cared for. And even more exciting: you can choose your favourite pattern from our wide range of fabrics and colours!

Introduction to the use of our products.

Persons with intellectual, reading or visual impairments must have our technical documentation read aloud or explained to them by a parent, carer or qualified technical advisor to ensure the safe use of our products.

Please note that only **spare parts and accessories** supplied by our company may be used and installed. This is essential to avoid compromising user safety. In individual cases, combinations with products from other manufacturers may be possible, but must first be assessed by a specialist dealer or by us as the manufacturer.

Settings and adjustments to the product or its accessories may only be carried out by persons who have received appropriate instruction from a medical device consultant. When making any adjustments, care must be taken to ensure that no limbs of the user or operator are within the adjustable range of movement in order to minimise the risk of injury.

For transport, the product must be adjusted to its most compact configuration. For example, set the seat, armrests and backrest to their lowest positions. Remove components that can be easily detached, such as the wheels on the **todd.** standing trainer or the handlebars on the **momo tricycle**. On products equipped with footrests, fold these up before transport.

For detailed information, please refer to the user manual for the specific product.

Instructions for use for each product are available at www.schuchmann.de/mediathek.

Where applicable, **product-specific cleaning instructions** in the respective user manuals must also be followed.

Cleaning measures according to material type.

Our products should be maintained and cleaned at regular intervals, to ensure that they remain functional throughout their service life. Choosing the right cleaning agents is crucial in this regard.

Wood: Wooden parts should be cleaned with a damp cloth or similar. Please follow the manufacturer's instructions regarding the concentration of the cleaning agent. Strong household cleaners or abrasive substances are not suitable. Never leave wood, a natural product, damp or wet for long periods of time, as it can absorb water and change shape (swelling). Therefore, always wipe and care for it with a damp cloth (e.g. ly with a wrung-out cloth). Test for compatibility on an inconspicuous area.

Powder-coated parts: Aluminium parts should be cleaned using a sponge or damp cloth with standard household cleaning agents.

Stainless steel: Stainless steel parts should be cleaned using a sponge or damp cloth with mild soapy water. Bleach must not be used.

Covers / Fabric parts: Covers and fabric parts should be washed in the washing machine at 40°C using standard washing powder and must not be tumble dried.

Plastics: Plastics should be cleaned using a sponge or damp cloth with mild soapy water. Bleach must not be used.

PU (polyurethane): PU parts should be cleaned using a sponge or damp cloth with mild soapy water. Bleach must not be used.

If you have any questions,
please contact us
directly at!



Disinfection measures according to material type.

Various products can be used to **disinfect metal, wood and plastic surfaces**. The disinfectants used should be **broadly effective** (at least bactericidal and yeasticidal, virucidal if necessary) and approved for use on medical devices. Suitable disinfectants can be found in the VAH disinfectant list. Care must be taken to ensure that disinfectants are used which are themselves medical devices.

Other disinfectants are not approved for disinfecting medical devices.

Liquid disinfectants are available as ready-to-use solutions that can be sprayed on and applied evenly with a soft cloth. Alternatively, **cloths pre-soaked in disinfectant** can be used to wipe down the entire surface of the products. In both cases, care must be taken to ensure that the surface is completely covered. The exposure times may vary and can be found in the manufacturer's instructions for the products used. However, no alcohol-based products should be used.

Airion systems may be used for occupational safety purposes. Contaminated equipment is placed inside the system and disinfected using cold fogging. Methods of disinfection using airion freezing are not permitted.

Where applicable, **product-specific disinfection instructions** from the respective instructions for use must be observed.

Hygiene measures for all products.

1. Who should carry out the hygiene measures?

Users or accompanying persons themselves, service employees of the specialist retailers and other authorised institutions.

2. And how often?

- Cleaning: Regularly and as needed (when dirty)
- Cleaning and disinfection: Regularly and as required (in the event of possible contamination with and prior to maintenance and servicing measures)
- After cleaning and disinfection: Damaged bases or components must be removed or repaired.
- Before reuse: reprocessing by trained personnel (thorough cleaning and disinfection). To ensure adequate restoring force of the padding and thus the product's performance, all padding components should be replaced at least every two years, preferably as part of reconditioning prior to reuse. Fabric covers must also be replaced before reuse to ensure a hygienically and technically sound condition. Only if a validated and reliable process is in place that guarantees hygienic reprocessing of the covers, and all covers are in technically flawless condition, is replacement of the covers prior to reuse not strictly necessary.

3. What should you use for this?

Wearing **protective gloves** (e.g. made of nitrile or butyl rubber) is recommended! Avoid liquids getting into Bowden cables, frame tubes, etc. Remove visible dirt with a damp sponge cloth and, if necessary, a cleaning agent. Do not use abrasive agents or tools!

4. How exactly do you go about doing this?

To clean, wipe all accessible surfaces with a damp sponge cloth and, if necessary, a suitable cleaning agent. Then dry the surfaces with a cotton or disposable cloth. Hard-to-reach areas can also be dried with a hairdryer set to a low temperature. The use of high-pressure cleaners is not permitted, as this can damage the products. For disinfection, the disinfectant used should be applied according to the manufacturer's instructions. Ensure **complete wetting**. If necessary, remove excess product with an uncoloured disposable cloth. It is particularly important to observe the manufacturer's recommended application times!

