

basti.



Instructions for use.

basti. The versatile bathing and shower cradle.




schuchmann®

Many thanks.



Dear Customer

At this point we would like to thank you for placing your trust in our company and for purchasing our product. We ask you to read through the Instructions for use carefully prior to initial commissioning of the product, and to observe them. Please note that guidelines and representations in these Instructions for use may deviate from your product due to differing equipment. We reserve the right to make technical modifications.

Important information!

Ensure that these Instructions for use remain with the product.

Your **schuchmann** Team



1. Preparation.	05
1.1 Delivery	05
1.2 Safety measures prior to use	05
1.3 Safe disposal	05
1.3.1 Packaging	05
1.3.2 Product	05
1.4 Where to store the Instructions for use	05
2. Product description.	06
2.1 Material information	06
2.2 Transport / handling	06
2.3 Application areas, use according to the intended purpose	06
2.3.1 Indication	06
2.3.2 Contraindications	06
2.4 Use not in accordance with the intended purpose / warning guidelines	07
2.5 Equipment for basic model	08
2.6 List of accessories	08
2.7 Product overview	08
3. Settings.	09
3.1 Backrest	09
3.2 Headrest	09
3.3 Calf support	09
3.4 Chest and leg strap	10
4. Accessories.	10
4.1 Head guide pad	10
4.2 Additional cover	10
4.3 Additional retaining strap	11
4.4 Shower cradle sub-frame	11
4.5 Sub-frame settings	12
5. Cleaning and maintaining.	14
5.1 Cleaning and disinfecting	14
5.1.1 Cleaning	14
5.1.2 Disinfection	14
5.2 Servicing	14
5.3 Maintenance	14
5.4 Spare parts	15
5.5 Duration of use and re-use	15
6. Technical data.	16
6.1 Dimensions bathing and shower cradle	16
6.2 Dimensions sub-frame	17



7. Guarantee. 17

8. Identification. 18

8.1 EU Declaration of Conformity..... 18

8.2 Serial number / date of manufacture..... 19

8.3 Product version 19

8.4 Issue of the document 19

8.5 Name and address of the manufacturer, specialist dealer sup-
plying the product..... 19

1. Preparation.

1.1 Delivery

On receiving the product, please check it for completeness, lack of faults and any transport damage. Inspect the goods in the presence of your forwarder. Should transport damage have occurred, please arrange for an inventory (determination of the faults) to be made in the presence of the forwarder. Please send a complaint in writing to the specialist dealer responsible.

1.2 Safety measures prior to use

The correct use of the product requires detailed and thorough instruction of the user or the accompanying person. We ask you to read through the Instructions for use carefully prior to initial commissioning of the product, and to observe them. It is possible that product parts that get in contact with the skin may heat up in the sun. Depending on the duration and intensity of solar radiation, the surfaces of individual parts can heat up to over 41°C and thus lead to slight burns in the event of direct skin contact. Therefore cover these areas or protect the device from direct sunlight.

1.3 Safe disposal

In order to preserve and protect the environment, to prevent environmental pollution and to improve the recycling of raw materials, please note the disposal instructions in **points 1.3.1 and 1.3.2.**

1.3.1 Packaging

The packaging of the product should be kept for any future transport that might be required. Should you have to return the product for repairs or in case of a guarantee claim, please if possible use the original box so that the product is optimally packaged. Otherwise, separate the packaging materials for recycling according to their classification.



Do not leave packaging materials unattended, as they are a possible source of danger.

1.3.2 Product

At the end of the product life cycle, recycle the raw materials used in the product according to their nature (see material information under **point 2.1).**

1.4 Where to store the Instructions for use

Please store these Instructions for use carefully and ensure that these Instructions for use remain with the product in case of re-use. Should you lose the instructions, you can always download an updated version at www.schuchmann.de.



2. Product description.

2.1 Material information

The stainless steel main frame is powder-coated and free from corrosion. The frames of the backrest and headrest as well as the calf support is made of aluminium, are black powder-coated and free from corrosion. The internal structure of the ratchet joints and the bowden cables are made from stainless steel and are free from corrosion. The removable and washable net cover is made from 100% polyester.

2.2 Transport / handling

basti. can be handled without any problems due to its low dead weight. You can lift the bathing cradle at any point. To transport the bathing cradle, adjust the settings to the most compact dimensions.

2.3 Application areas, use according to the intended purpose

The **basti.** bathing and shower cradle is a medical product of risk class 1 and is suited exclusively for use in bathtubs, showers, in the sandbox and on the beach. It supports users who cannot sit unaided and offers additional hold thanks to retaining straps. Any other use or use over and above this purpose shall be considered not in accordance with the intended purpose.

2.3.1 Indication

The **basti.** bathing and shower cradle is suited for:

- Users who are unable to sit or lie down in the bathtub due to dysfunction of the lower limb with significantly reduced or complete failed joint mobility
- Users with limitation to the muscular strength of the lower extremities
- Users with damage to the nervous system, damage to neuromusculoskeletal functions
- Users with damage to the movement-related function and/or muscle function and/or with damage to the neurological function
- Users with movement patterns with significantly reduced or failed trunk control or reduced or failed body control

2.3.2 Contraindications

In general, the indication should be accompanied by a doctor/orthopaedist. It should therefore be clarified prior to procurement whether contraindications exist for the patient. In general, any type of pain represents a contraindication.

2. Product description.



2.4 Use not in accordance with the intended purpose / warning guidelines

- Ensure that the bathing and shower cradle is used by one user only.
- Never leave the user seated on the bathing and shower cradle without supervision.
- Correct use of the bathing and shower cradle requires precise and careful training of the accompanying person.
- The max. load (see **Point 6.1**) must not be exceeded.
- Do not use the bathing and shower cradle if it has defective, worn or missing parts.
- For reasons of fire safety, the bathing and shower cradle may not be placed close to an open fire or any other strong source of heat such as electric or gas heaters.
- Use only accessories and spare parts made by Schuchmann, otherwise you will compromise the safety of the user.
- Only use the bathing and shower cradle if all components have been correctly mounted and adjusted.
- The bathing and shower cradle may not be used to transport people.
- Users who have difficulty reading must have someone read these instructions for use aloud to that they understand how to use the bathing cradle.



2. Product description.

2.5 Equipment for basic model

- Frame, black powder coated
- removable and washable cover
- Backrest, knee area and head guide pad angle adjustment
- Chest strap

2.6 List of accessories

- Head guide pad, adjustable with hook and loop closure
- additional retaining strap, adjustable with hook and loop closure for the chest and pelvic area
- Sub-frame shower cradle, movable with 4 braked castors
- Cover with pelvic mould depression and adjustable seat support for ischiatic bone

2.7 Product overview

The Fig. below is intended to show you the designation of the most important components as well as the terms which you will find in these Instructions for use.




3. Settings.

basti. transforms any shower and bathing situation into child's play. Thanks to the backrest and knee angle adjustment, it can be put optimally to a desired position that is comfortable for the user. Thanks to the folding head part, washing the hair becomes effortless and due to the low seat height only a small quantity of water is needed when bathing.

3.1 Backrest

The backrest can be adjusted from 90° to 180°. For **basti.** size 1 to 3, pull on the loop (A) on the rear of the backrest and bring the backrest to the desired position. Release the loop after positioning and ensure that the ratchet joints have engaged.

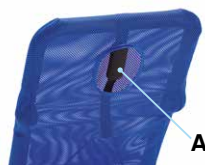


For size 4 release the two stainless steel ratchet joints (B), to adjust the backrest.




3.2 Headrest

The headrest (C) can be set from 0° to 90°. Press the pushbuttons (D) on both sides and bring the headrest to the desired position. After positioning release the pushbuttons and ensure that the ratchet joints have engaged.

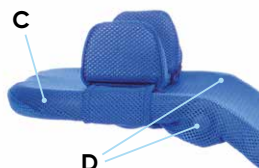

3.3 Calf support

The calf support (E) can be adjusted between 0° and 90°. Press the pushbuttons (F) on both sides and bring the calf support to the desired position. After positioning release the pushbuttons and ensure that the ratchet joints have engaged.

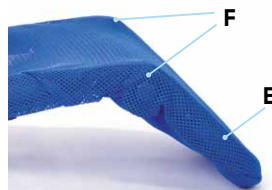

The ratchet joints must engage after each adjustment!

3. Settings.

3.4 Chest and leg strap

To attach the chest and leg strap (A), loop it around the bathing cradle at the desired position and fix it from the rear using the hook and loop closure. The width of the strap can be adjusted using additional hook and loop closures on the sides.



4. Accessories.

Settings and adjustments to the product or accessories may only be made by people who have been given the necessary instructions by a medical product advisor. Please ensure that none of the user's extremities are in the respective area when making adjustments of any kind to minimise the risk of injury.

4.1 Head guide pad

- adjustable
- with hook and loop closure

Attachment

To attach the head guide pad (B), loop the strap around the headrest at the desired position and fix it from the rear using the hook and loop closures. The two head guide pads can be adjusted horizontally.



4.2 Additional cover

- with pelvic mould depression
- with adjustable seat support for ischiatic bone
- with hook and loop closure

4. Accessories.

The pelvic mould depression (**A**) also stabilises the seating position of smaller users and forms the basis for upright seating. Moreover, the adjustable seat support for ischiatic bone (**B**) defines the seating surface.

Attachment

Bring the backrest in a position almost parallel to the seat surface and pull the cover over the frame. Fix it using the hook and loop closure below the seat surface.




Make sure that the hook and loop closures are closed!

4.3 Additional retaining strap

- adjustable
- with hook and loop closure
- for the chest or pelvic area

Attachment

To attach the additional retaining strap (**A**), loop it around the frame at the desired position and fix it using the hook and loop closure.

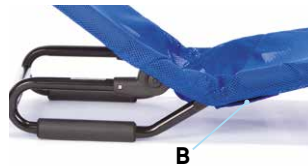

4.4 Shower cradle sub-frame

- movable
- with 4 braked castors
- Total width: 57 cm
- Lying height: 61* – 66 cm
- Total length: 92 cm


* In combination with the cover with pelvic mould depression

4. Accessories.

Attachment on the sub-frame

Insert the rear crossbar (A) on the bathing cradle first into the rear mounts (B) and then into the latch-in mounts of the sub-frame (C). Move the bathing cradle and make sure that it has correctly engaged in the sub-frame. The width of the mounts (B+C) can be adjusted and can thus be adjusted to the individual size (see **Point 4.5**).

To take out the **basti.** bathing cradle, simply pull on the pull strap (D) and lift off the bathing cradle with the other hand.

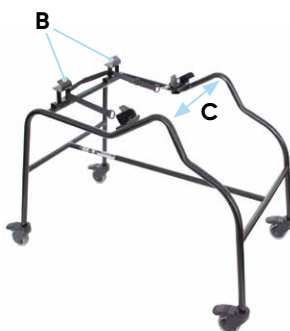

Make sure that the hook and loop closures are closed!




4.5 Sub-frame settings

The rear mounts (E) and the latch-in mounts (F) must be fixed at different positions for the respective sizes of **basti.**

basti. bathing and shower cradle settings Size 1


For size 1, put the mounts (E) and the latch-in mounts (F) in the following positions:

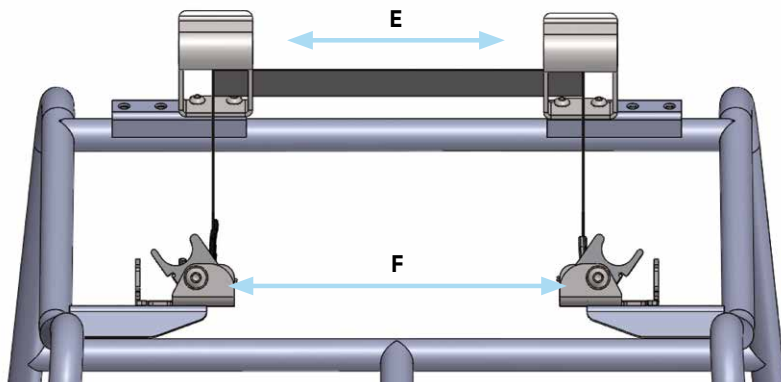

4. Accessories.


basti. bathing and shower cradle settings Size 2 + 3

For sizes 2+3 put the mounts (A) and the latch-in mounts (B) in the following positions:

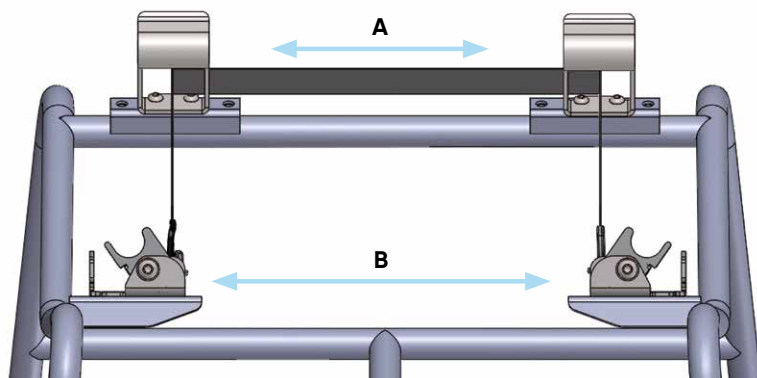

basti. bathing and shower cradle settings Size 4

For size 4, put the mounts (C) and the latch-in mounts (D) in the following positions:

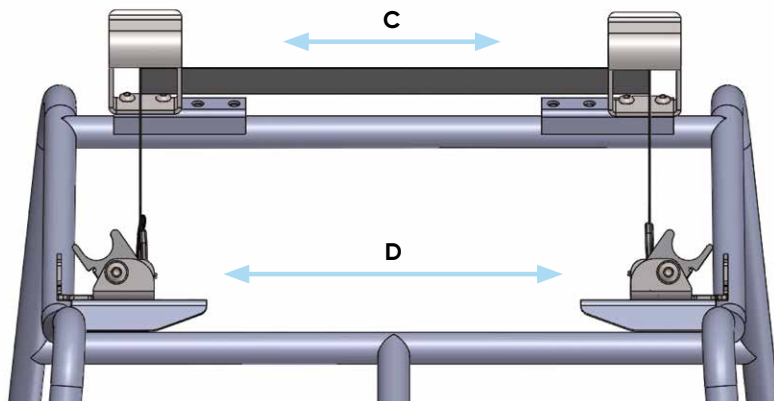

5. Cleaning and maintaining.

5.1 Cleaning and disinfecting

5.1.1 Cleaning

Please clean all frame elements regularly with a sponge or damp cloth; making sure that water droplets are removed. Please clean with a mild household detergent for severer dirt. Thorough drying of the cleaned areas is important.

All fabrics that cannot be removed can be wiped with a moist cloth. For all removable fabrics please pay attention to the sewn-in care labels (such as **A + B**) on the respective element.

Please also pay attention to our general cleaning and hygiene advice. This can be found at www.schuchmann.de/mediathek.



5.1.2 Disinfection

Various products can be used for surface disinfection of metal and plastic parts.

Liquid disinfectants are available as ready-to-use solutions that are sprayed on and evenly applied with a soft cloth. Alternatively, wipes pre-soaked with disinfectant can be used to wipe the products over the entire surface. In both cases, care must be taken to ensure complete wetting. Disinfection in fully automatic disinfection systems is also possible and recommended. The exposure times may vary and can be found in the manufacturer's instructions for the products used.

5.2 Servicing

Please carry out a visual inspection on a daily basis and regularly check the product for cracks, breaks, missing parts and malfunctions. In case of a defect or malfunction, please contact the specialist dealer who supplied you with the product (see **Point 8.5**).

5.3 Maintenance

The product does not require maintenance.

5. Cleaning and maintaining.

5.4 Spare parts

Use only original accessories and original spare parts from Schuchmann, as otherwise the safety of the user is endangered and the warranty expires.

To order spare parts, please contact the supplying specialist dealer (see **point 8.5**) stating the serial number of the product. Necessary spare parts and accessories must only be installed by trained personnel.

5.5 Duration of use and re-use

The expected duration of use of our product, dependent on the usage intensity and amount of re-use, totals up to "8" years if the usage takes place in accordance with the information in these Instructions for use. It may be possible to use the product over and above this time period if it is in a safe condition. The expected duration of use does not refer to wear parts, such as for example wheels, gas pressure springs,... . The maintenance and evaluation of the condition, and if applicable the potential for re-use, must be decided by the specialist dealer.

The product is suited for re-use. Prior to forwarding, please follow the cleaning and disinfection instructions stated in **Point 5.1**. Accompanying documents such as these Instructions for use are part of the product and must be passed on to the new user. No disassembly is required prior to re-use. In the case of storage, it is recommended to fold the product to the smallest dimension to save space.




Should a serious incident occur during the service life of the product despite being used as intended, this must be reported immediately to the manufacturer and the competent authority.



6. Technical data.


6.1 Dimensions bathing and shower cradle

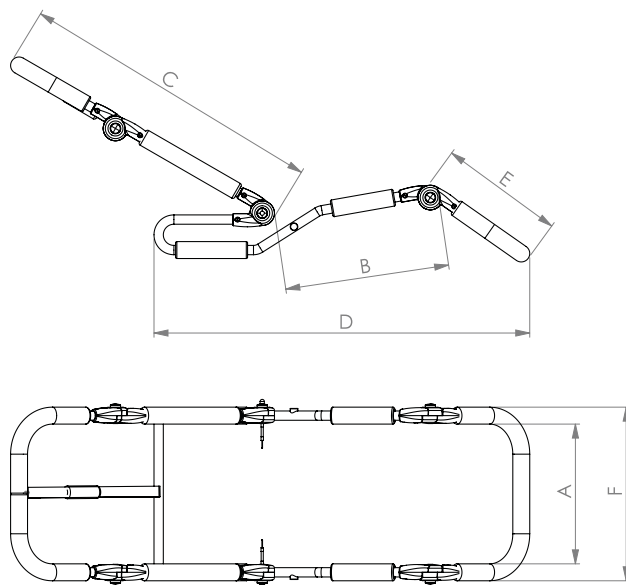

		Size 1	Size 2	Size 3	Size 4
A	Seat width	30 cm	35 cm	35 cm	40 cm
B	Seat depth	30 cm	34 cm	38 cm	44 cm
C	Back height	45 cm	61 cm	71 cm	82 cm
D	Standing space length	69 cm	74 cm	88 cm	93 cm
E	Lower leg length	19 cm	20 cm	28 cm	28 cm
F	Total width	36 cm	41 cm	41 cm	46 cm
	Back angle	90° - 180°	90° - 180°	90° - 180°	90° - 180°
	Knee angle	0° - 90°	0° - 90°	0° - 90°	0° - 90°
	Headrest angle	0° - 90°	0° - 90°	0° - 90°	0° - 90°
	Seat height	0* / 5 cm	0* / 5 cm	0* / 5 cm	0* / 5 cm
	max. load	35 kg	45 kg	55 kg	75 kg
	Weight	4 kg	4.1 kg	4.3 kg	4.8 kg
	* In combination with the cover with pelvic mould depression				

6. Technical data

6.2 Dimensions sub-frame

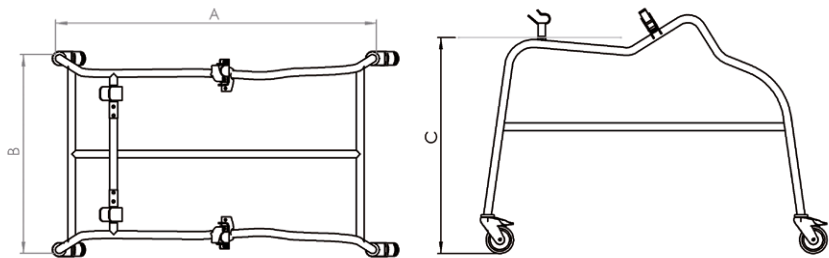

A	Length	92 cm
B	Width	57 cm
C	Seat height with sub-frame	61* / 66 cm
	* In combination with the cover with pelvic mould depression	

7. Guarantee.

The two-year statutory guarantee period shall apply for all products. It starts with the delivery or handover of the goods. Should a verifiable material or manufacturing fault occur within this time period, we shall, after carriage paid return to us, view the indicated damage and, if applicable, either repair or deliver a new product at our discretion.



8. Identification.

8.1 EU Declaration of Conformity



EU Konformitätserklärung

EU Declaration of Conformity



Firma / Company Schuchmann GmbH & Co. KG
Rudolf-Runge-Str. 3 · 49143 Bissendorf · Deutschland / Germany
Tel. +49 (0) 5402 / 40 71 00 · Fax +49 (0) 5402 / 40 71 109

erklärt in alleiniger Verantwortung, dass das nachfolgend genannte Produkt der Risikoklasse 1
declares under our sole responsibility that the following product(s) of Class 1 Medical Devices

„basti.“ Bade- und Duschieliege / *bathing and shower bench*

Art.-Nr. / Item-No.: 56 01 000, 56 02 000, 56 03 000, 56 04 000

Basis UDI-DI / Basic UDI-DI: 4251040200004000560XXXXBV

den einschlägigen Bestimmungen der im folgenden aufgeführten Richtlinien und Standards entspricht:
is / are in conformity with the requirements of the below listed directives and standards:

Verordnung (EU) 2017/745 über Medizinprodukte vom 05. April 2017
Regulation (EU) 2017/745 on medical devices of 5 April 2017

DIN EN 12182:2012 Technische Hilfen für behinderte Menschen
Technical aids for disabled persons

DIN EN ISO 14971:2013 Medizinprodukte – Anwendung des Risikomanagements auf Medizinprodukte
Medical devices – Application of risk management to medical devices

Diese Konformitätserklärung gilt nur für Produkte mit den oben genannten Artikelnummern und
ist gültig bis zum 31.12.2023.

*This declaration of conformity applies only for products with above-named Item-numbers
and is valid until 31.12.2023.*

Datum / Date: 01.04.2020

Unterschrift / Sign:



Name / Name: Torsten Schuchmann

Funktion / Function: Sicherheitsbeauftragter für Medizinprodukte / *Safety officer for medical devices*

8. Identification.

8.2 Serial number / date of manufacture

The serial number, the manufacturer date as well as other details can be found on the type plate that can be found on each of our products (A = bathing cradle, B = sub-frame).

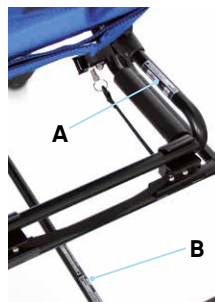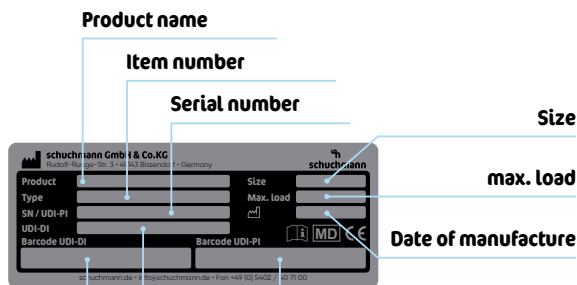

Product labelling in accordance with MDR

8.3 Product version

The **basti.** bathing and shower cradle is available in four sizes and can be complemented with various accessories (see **Point 4**).

8.4 Issue of the document

Instructions for use **basti.** – Change status J; issue 03.2021

8.5 Name and address of the manufacturer, specialist dealer supplying the product

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
This product was manufactured by:



Schuchmann GmbH & Co. KG

Rudolf-Runge-Str. 3 · 49143 Bissendorf · Germany
Phone +49 (0) 5402 / 40 71 00 · Fax +49 (0) 5402 / 40 71 109
info@schuchmann.de · www.schuchmann.de

This product has been delivered by the following specialist dealer:





schuchmann.de