

bibi.



Instructions for use.

bibi. The lightweight and compact bath seat.




schuchmann®

Many thanks.



Dear Customer

At this point we would like to thank you for placing your trust in our company and for purchasing our product. We ask you to read through the Instructions for use carefully prior to initial commissioning of the product, and to observe them. Please note that guidelines and representations in these Instructions for use may deviate from your product due to differing equipment. We reserve the right to make technical modifications.

Important information!

Ensure that these Instructions for use remain with the product.

Your **schuchmann** Team



1. Preparation.	04
1.1 Delivery	04
1.2 Safety measures prior to use	04
1.3 Safe disposal	04
1.3.1 Packaging	04
1.3.2 Product	04
1.4 Where to store the instructions for use	04
2. Product description.	05
2.1 Material information	05
2.2 Transport / handling	05
2.3 Application areas, use according to the intended purpose	05
2.3.1 Indication	05
2.3.2 Contraindications	05
2.4 Use not in accordance with the intended purpose / warning guidelines	06
2.5 Equipment for basic model	06
2.6 List of accessories	06
2.7 Product overview	06
3. Settings.	07
3.1 Back angle	07
3.2 Cover	07
4. Cleaning and maintaining.	08
4.1 Cleaning and disinfecting	08
4.1.1 Cleaning	08
4.1.2 Disinfection	08
4.2 Servicing	08
4.3 Maintenance	08
4.4 Spare parts	09
4.5 Duration of use and re-use	09
5. Technical data.	10
6. Guarantee.	10
7. Identification.	11
7.1 EU Declaration of Conformity	11
7.2 Serial number / date of manufacture	12
7.3 Product version	12
7.4 Issue of the document	12
7.5 Name and address of the manufacturer, specialist dealer supplying the product	12



1. Preparation.

1.1 Delivery

On receiving the product, please check it for completeness, lack of faults and any transport damage. Inspect the goods in the presence of your forwarder. Should transport damage have occurred, please arrange for an inventory (determination of the faults) to be made in the presence of the forwarder. Please send a complaint in writing to the specialist dealer responsible.

1.2 Safety measures prior to use

The correct use of the product requires detailed and thorough instruction of the user or the accompanying person. We ask you to read through the Instructions for use carefully prior to initial commissioning of the product, and to observe them. It is possible that product parts that get in contact with the skin may heat up in the sun. Depending on the duration and intensity of solar radiation, the surfaces of individual parts can heat up to over 41°C and thus lead to slight burns in the event of direct skin contact. Therefore cover these areas or protect the device from direct sunlight.

1.3 Safe disposal

In order to preserve and protect the environment, to prevent environmental pollution and to improve the recycling of raw materials, please note the disposal instructions in **points 1.3.1 and 1.3.2.**

1.3.1 Packaging

The packaging of the product should be kept for any future transport that might be required. Should you have to return the product for repairs or in case of a guarantee claim, please if possible use the original box so that the product is optimally packaged. Otherwise, separate the packaging materials for recycling according to their classification.



Do not leave packaging materials unattended, as they are a possible source of danger.

1.3.2 Product

At the end of the product life cycle, recycle the raw materials used in the product according to their nature (see material information under **point 2.1**).

1.4 Where to store the instructions for use

Please store these Instructions for use carefully and ensure that these instructions for use remain with the product in case of re-use. Should you lose the instructions, you can always download an updated version at www.schuchmann.de.

2. Product description.

2.1 Material information

The frame is made from aluminium, free from corrosion and powder-coated in black. The internal structure of the ratchet joints is made of stainless steel and is free from corrosion. The removable and washable net cover is made from 100% polyester.

2.2 Transport / handling

Due to the low weight, **bibi.** can be handled with ease. You can lift the bath seat at any point. To transport the bath seat, adjust the settings to the most compact dimensions.

2.3 Application areas, use according to the intended purpose

The **bibi.** mobile bath seat is a medical product of risk class 1 and is suited exclusively for use in bathtubs, showers, in the sandbox and on the beach. It supports the user who cannot sit freely and offers additional support thanks to retaining straps. Any other use or use over and above this purpose shall be considered not in accordance with the intended purpose.

2.3.1 Indication

The **bibi.** bath seat is suitable for:

- Users who are unable to sit or lie down in the bathtub due to dysfunction of the lower limb with significantly reduced or complete failed joint mobility
- Users with limitation to the muscular strength of the lower extremities
- Users with damage to the nervous system, damage to neuromusculoskeletal functions
- Users with damage to the movement-related function and/or muscle function and/or with damage to the neurological function
- Users with movement patterns with significantly reduced or failed trunk control or reduced or failed body control

2.3.2 Contraindications

In general, the indication should be accompanied by a doctor/orthopaedist. It should therefore be clarified prior to procurement whether contraindications exist for the patient. In general, any type of pain represents a contraindication.



2. Product description.



2.4 Use not in accordance with the intended purpose / warning guidelines

- Make sure that the bath seat is used by only one user.
- Never leave the user unattended in the bath seat.
- The correct use of the bath seat requires precise and careful training of the attending caregiver.
- The max. load of 30 kg must not be exceeded.
- Do not use the bath seat with defective, worn or missing parts.
- For fire safety reasons, the bath seat must not be placed near an open fire or other strong heat source such as an electric or gas heater.
- Use only accessories and spare parts made by Schuchmann, otherwise you will compromise the safety of the user.
- Use the bath seat only if all components have been correctly fitted and adjusted.
- The bath seat is not suitable to transport persons.
- Users with reading difficulties must have the instructions read to them to understand handling the bath seat.

2.5 Equipment for basic model

- Aluminium frame with removable cover
- Backrest can be adjusted at an angle
- incl. lap belt

2.6 List of accessories

- additional lap belt

2.7 Product overview

The fig. below is intended to show you the designation of the most important components as well as the terms which you will find in these Instructions for use.

Backrest

Back angle adjustment

**Frame, black
powder-coated**



Lap belt


Seat

3. Settings.

Settings and adjustments to the product or accessories may only be made by people who have been given the necessary instructions by a medical product advisor. Please ensure that none of the user's extremities are in the respective area when making adjustments of any kind to minimise the risk of injury.

3.1 Back angle

The backrest can be adjusted at an angle. To this end, press the two ratchet joints (A) simultaneously and put the backrest in the desired position. After the positioning, release the ratchet joints and ensure that the ratchet joints have engaged.

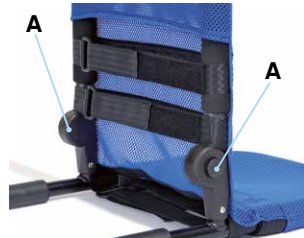

bibi. can be folded in a space-saving manner for transport.



The ratchet joints must engage after each adjustment!




3.2 Cover

The cover can be adjusted via the belt loops in the back or seat area. To this end, you can use two belt loops for the back area (B) and two for the seat area (C).



Make sure that the hook and loop closures are closed!

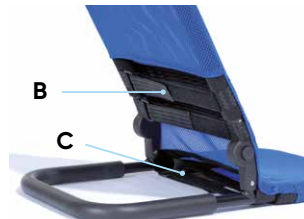

4. Cleaning and maintaining.

4.1 Cleaning and disinfecting

4.1.1 Cleaning

Please clean all frame elements regularly with a sponge or damp cloth; making sure that water droplets are removed. Please clean with a mild household detergent for severer dirt. Thorough drying of the cleaned areas is important.

All fabrics that cannot be removed can be wiped with a moist cloth. For all removable fabrics please pay attention to the sewn-in care labels (such as **A + B**) on the respective element.

Please also pay attention to our general cleaning and hygiene advice. This can be found at www.schuchmann.de/mediathek.



4.1.2 Disinfection

Various products can be used for surface disinfection of metal and plastic parts.

Liquid disinfectants are available as ready-to-use solutions that are sprayed on and evenly applied with a soft cloth. Alternatively, wipes pre-soaked with disinfectant can be used to wipe the products over the entire surface. In both cases, care must be taken to ensure complete wetting. Disinfection in fully automatic disinfection systems is also possible and recommended. The exposure times may vary and can be found in the manufacturer's instructions for the products used.

4.2 Servicing

Please carry out a visual inspection on a daily basis and regularly check the product for cracks, breaks, missing parts and malfunctions. In case of a defect or malfunction, please contact the specialist dealer who supplied you with the product (see **Point 7.5**).

4.3 Maintenance

The product does not require maintenance.

4. Cleaning and maintaining.

4.4 Spare parts

Use only original accessories and original spare parts from Schuchmann, as otherwise the safety of the user is endangered and the warranty expires.

To order spare parts, please contact the supplying specialist dealer (see **point 7.5**) stating the serial number of the product. Necessary spare parts and accessories must only be installed by trained personnel.

4.5 Duration of use and re-use

The expected duration of use of our product, dependent on the usage intensity and amount of re-use, totals up to "8" years if the usage takes place in accordance with the information in these Instructions for use. It may be possible to use the product over and above this time period if it is in a safe condition. The expected duration of use does not refer to wear parts, such as for example wheels, gas pressure springs,... . The maintenance and evaluation of the condition, and if applicable the potential for re-use, must be decided by the specialist dealer.

The product is suited for re-use. Prior to forwarding, please follow the cleaning and disinfection instructions stated in **point 4.1**. Accompanying documents such as these Instructions for use are part of the product and must be passed on to the new user. No disassembly is required prior to re-use. In the case of storage, it is recommended to fold the product to the smallest dimension to save space.




Should a serious incident occur during the service life of the product despite being used as intended, this must be reported immediately to the manufacturer and the competent authority.



5. Technical data.

Dimensions

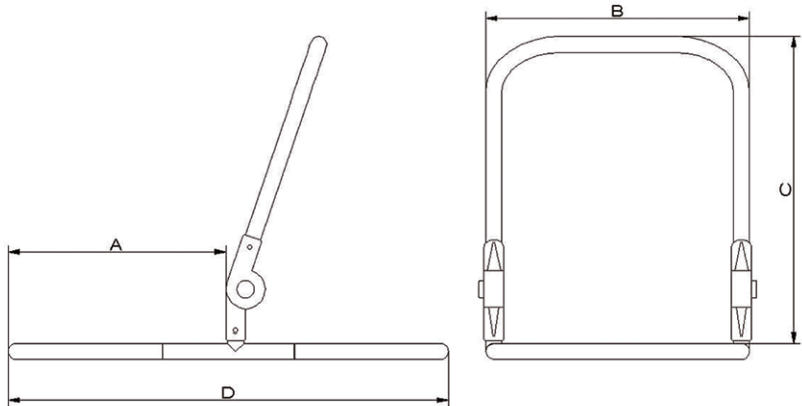

		bibi.	bibi. with long backrest
A	Seat depth	30 cm	30 cm
B	Seat width	36 cm	36 cm
C	High back	45 cm	55 cm
D	Total length	60 cm	60 cm
	Back angle	90° - 180°	90° - 180°
	max. load	30 kg	30 kg
	Weight	2.3 kg	2.3 kg

6. Guarantee.

The two-year statutory guarantee period shall apply for all products. It starts with the delivery or handover of the goods. Should a verifiable material or manufacturing fault occur within this time period, we shall, after carriage paid return to us, view the indicated damage and, if applicable, either repair or deliver a new product at our discretion.

7. Identification.

7.1 EU Declaration of Conformity



EU Konformitätserklärung EU Declaration of Conformity



Firma / Company Schuchmann GmbH & Co. KG
Rudolf-Runge-Str. 3 · 49143 Bissendorf · Deutschland / Germany
Tel. +49 (0) 5402 / 40 71 00 · Fax +49 (0) 5402 / 40 71 109

erklärt in alleiniger Verantwortung, dass das nachfolgend genannte Produkt der Risikoklasse 1
declares under our sole responsibility that the following product(s) of Class 1 Medical Devices

„bibi.“ Badesitz / bathing seat

Art.-Nr. / Item-No.: 55 01 000, 56 01 001, 55 01 002

Basis UDI-DI / Basic UDI-DI: 4251040200004000550XXXXBG

den einschlägigen Bestimmungen der im folgenden aufgeführten Richtlinien und Standards entspricht:
is / are in conformity with the requirements of the below listed directives and standards:

Verordnung (EU) 2017/745 über Medizinprodukte vom 05. April 2017
Regulation (EU) 2017/745 on medical devices of 5 April 2017

DIN EN 12182:2012 Technische Hilfen für behinderte Menschen
Technical aids for disabled persons

DIN EN ISO 14971:2013 Medizinprodukte – Anwendung des Risikomanagements auf Medizinprodukte
Medical devices – Application of risk management to medical devices

Diese Konformitätserklärung gilt nur für Produkte mit den oben genannten Artikelnummern und
ist gültig bis zum 31.12.2023.

*This declaration of conformity applies only for products with above-named Item-numbers
and is valid until 31.12.2023.*

Datum / Date: 01.04.2020

Unterschrift / Sign:

Name / Name: Torsten Schuchmann


Funktion / Function: Sicherheitsbeauftragter für Medizinprodukte / *Safety officer for medical devices*



7. Identification.

7.2 Serial number / date of manufacture

The serial number, the date of manufacture and other information can be found on the type plate, which is located on all of our products (**A**).

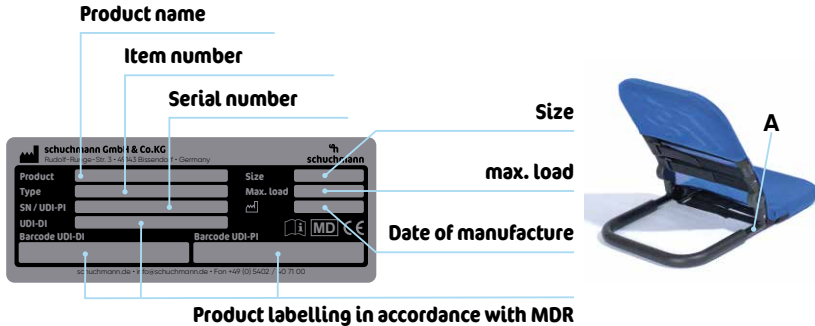

7.3 Product version

The bath seat **bibi.** is available in two sizes and can be complemented by accessories (see **point 2.6**).

7.4 Issue of the document

Instructions for use **bibi.** – Change status H; issue 03.2021

7.5 Name and address of the manufacturer, specialist dealer supplying the product

This product was manufactured by:



Schuchmann GmbH & Co. KG

Rudolf-Runge-Str. 3 · 49143 Bissendorf · Germany
 Phone +49 (0) 5402 / 40 71 00 · Fax +49 (0) 5402 / 40 71 109
 info@schuchmann.de · www.schuchmann.de

This product has been delivered by the following specialist dealer: